



Hospitals



# Hospitals

We've worked with state governments and large, well-known private healthcare groups. Our experience ranges from assisting individual departments to source single items, to complete fit outs of new hospital builds.



I-move



I-move Chart



I-move Drawer



I-move Slim



Contour Recline



Contour Recline Endo



Contour Recline E-Vertex



Contour Recline EEG



Contour Recline Barituff




Chair Oxy Holder



Chair IV Pole



Chair Foot Board

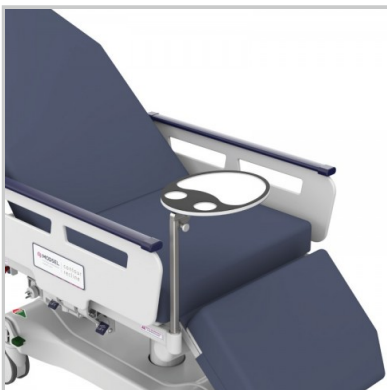

Chair Meal Tray



Chair Arm Board



Chair Padded Armrests



Contour Energy



Contour Portare



Contour E-Vertex



Contour Conceal



Contour Jazz



Contour Classic



Contour Exami



Contour Barituff



Contour Deluxe



Contour Heli-Transfer



Contour Portare-X



Contour Multi-X




Contour Barituff-X




Contour Endosurge



Contour Endo-X

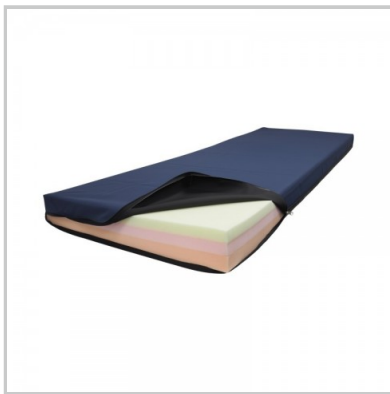

Stretcher Mattress

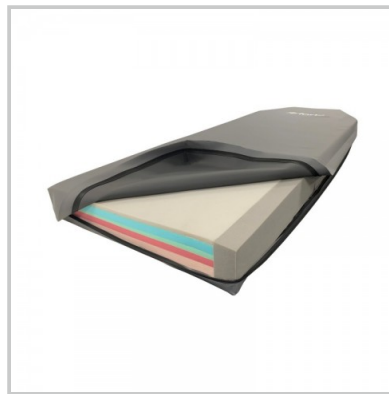

Premium Stretcher  
Mattress



Orbit-Drive





Dropside Options



Stretcher IV Poles



Oxy Holders



Dropdown Push Handles



Document Holder



End Rail



End Board



Dropdown Foot Board



Folding Monitor Board



Folding Monitor-Foot Board



Weigh Scales



Stretcher Colour Options



Dropside Padding



Meal Tray



Aquatuff Electric



Change Table

# I-move

A height adjustable rounds trolley with storage that is light and easy to move in tight spaces.



## Key Benefits

**Compact & simple:** Light, easy to move and unobtrusive in tight spaces is the perfect partner to make sure your nursing rounds are efficient and easy.

**Tech support:** Upgrade your I-move to support any and all of your IT requirements required for the job.

**Safe & secure:** Option to add a Kensington lock to restrain valuable laptop hardware to keep safe as well as cable management options.

## Why Buy?

The i-move is an affordable and height adjustable rounds trolley. When it comes to moving laptops, tablets or simply pens and paper along with other clinical items, the only practical solution has been the traditional metal dressing trolley.

How good would it be to have a trolley that is light and easy to move, unobtrusive in tight spaces and that you can comfortably sit or stand at? Coupled with this is a convenient storage shelf and our ability to customise the top size and shape to totally suit your requirements!



# I-move



## Storage space

Convenient storage space with retaining lip across the front.



## Customisable

Use our standard work surface or customise to suit your specific requirements.



## Height adjustable

Simply squeeze the lever under the top and effortlessly adjust to your desired work height.



## Storage compartment

Add a removable stainless steel storage compartment for cables / pens etc.



## Power supply holder

Securely hold a laptop power supply for easy charging.



## Kensington lock

Restrain valuable laptop hardware to keep it safe.



## Add your IT

Add your own holders to suit the specific IT hardware in use.

## Specifications

**Size:** Compact, easy to clean top - 550mm wide x 560mm deep

**Height adjustable:** From 840mm to 1140mm

**Storage size:** Convenient 100mm storage space with lip

## Product Codes

I-Move Trolley; MCT-IM-LHA

Removable Storage Container - MCT-IMA-SC

Power Supply Holder - MCT-IMA-PCH

Kensington Lock - MCT-IMA-KL

# I-move Chart

A compact chart trolley to maximise your working space with effortless positioning.



## Key Benefits

**Effortless adjustment:** Simply squeeze the lever under the top and effortlessly adjust to your desired work height.

**Custom options:** Choose between retaining lip options to help retain items from sliding off or remove the push handles if not required for a slim finish, the choice is yours!

**Simple storage:** Generous storage space under the top allow you to store extra items in a safer manner.

## Why Buy?

With clips on all sides of the top and designed to hold two A3 sheets side by side, the i-move chart trolley secures charts from any position to ensure writing and reading is easy. The compact footprint of this trolley coupled with a height adjustable 700mm square top with storage, has been carefully considered to maximise your working space and help in quick, effortless positioning.

# I-move Chart



## Retaining edge

The push handles with retaining lip help to retain items from sliding off.



## Angled top

The top surface is sloped to a comfortable working angle.



## Storage compartment

There is generous storage for extra items under the top.



## Height adjustable

Simply squeeze the lever under the top and effortlessly adjust to your desired work height.



## Lip option

Remove the push handles if not required for a slim finish

## Specifications

**Size:** 520 x 710mm with safe rounded edges

**Height Range:** Gas assisted height adjustment from 800 - 1100mm (top of push handle)

**Designed For:** 4 x A4 sheets, 2 x A3 sheets or 1 x A2 sheet

**Castor Size:** 75mm diameter twin wheel castors (2 locking)

**Includes:** 4 x stainless steel spring clips, rear storage compartment under the top & retaining lip options across the bottom of the work surface

## Product Codes

I-Move Chart; MCT-IM-CT

Lip Option; MCT-IM-RL

# I-move Drawer

A height adjustable rounds trolley with a secure locking drawer.



## Key Benefits

**Safe & secure:** Quality metal lockable drawer with ball bearing runners allows you to safely store valuables and secure them.

**Custom design:** Choose between a standard work surface or customise to suit your desired requirements.

**Tech savvy:** Upgrade your I-move Drawer to support a wide range of IT hardware.

## Why Buy?

This i-move is a height adjustable rounds trolley with a secure locking drawer. When it comes to moving laptops, tablets or simply pens and paper along with other clinical items, the only practical solution has been the traditional metal dressing trolley.

How good would it be to have a trolley that is light and easy to move, unobtrusive in tight spaces and that you can comfortably sit or stand at? Coupled with this is a convenient storage drawer and our ability to customise the top size and shape to totally suit your requirements!

# I-move Drawer



## Lockable drawer

Safe 400 wide x 390 x 150mm deep drawer on quality ball bearing runners.



## Compact design

Move your items easily and safely with this practical rounds trolley.



## Customisable

Use our standard work surface or customise to suit your specific requirements.



## Height adjustable

Simply squeeze the lever under the top and effortlessly adjust to your desired work height.



## Accessories for IT

Add holders or locks for a wide range of IT hardware.

## Specifications

**Size:** Compact, easy to clean top - 550mm wide x 645mm deep

**Drawer Size:** 400 wide x 390 x 150mm deep drawer size

**Height Range:** Height adjustable from 930mm to 1230mm

**Options:** Laptop security lock and tablet mount systems available

## Product Code

MCT-IM-LHA-LD

# I-move Slim

No under top storage for an ultra lightweight, height adjustable and easy to move work surface.



## Key Benefits

**Compact & simple:** Lightweight design creates a practical rounds trolley for people 'on the go'

**IT savvy:** Add holders or locks for a wide range of IT hardware. Go to the I-Move IT trolley to see all options.

**Custom design:** Choose between a standard work surface or customise to suit your desired

## Why Buy?

How good would it be to have a height adjustable, compact work surface that is very light and easy to move in tight spaces and that you can sit or stand at? For any area of any healthcare facility, this trolley brings a very useful work surface with you, wherever you go.



# I-move Slim



## Ultra thin

A lightweight, practical rounds trolley for people 'on the go'.



## Customisable

Use our standard work surface or customise to suit your specific requirements.



## Height adjustable

Simply squeeze the lever under the top and effortlessly adjust to your desired work height.



## Accessories for IT

Add holders or locks for a wide range of IT hardware. Go to the I-Move IT trolley to see all options.

## Specifications

**Size:** Compact, easy to clean top - 550mm wide x 560mm deep

**Height Range:** Height adjustable from 730mm to 1030mm

**Options:** Laptop security lock system, tablet mount & rear cable storage compartment options

## Product Code

MCT-IM-LHA-ST

# Contour Recline

A versatile chair to streamline medical examination, procedures and recovery across many healthcare facilities.



## Key Benefits

**Save Space:** Fit more chairs in the same space by increasing throughput with faster recoveries from patients sitting up right.

**Hygiene conscious:** Smooth, gap free platform aids cleaning and helps prevent fluid ingress to meet the highest infection control standards.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with quality, care and customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance and fast guaranteed parts.

## Application

Ideal for Day Surgery, Recovery, Transit Lounge, Fast Track ED & Cardiac

## Why Buy?

Make better use of your valuable space and streamline your department with Modsel's compact procedure chairs. They are easy to move in and out of tight areas meaning a higher patient throughput and reduced waiting times. Patients can be positioned easily from sitting upright to a full recline, or lay completely flat with the stability of twin columns under the chair which can be used for CPR or specialised procedures. We've taken care with every aspect of design to ensure comfort and efficiency for both the patient and carer. Pop-up push handles ensure best operator positioning while preventing transfer movement to the patient.

# Contour Recline



## Low and safe

Excellent 460mm low height with fold-under drop-sides.

The ultra-low height is combined with drop-sides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patient's legs or clothes.



## Easy egress

Forward tilt facilitates getting on and off.

For aged or disabled patients, a gentle incline and the long drop-side armrests may assist in safer transfer.



## Lies flat

The backrest and leg rest move independently or together from chair to bed.

You now have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors. The unique handle design correctly positions the carer while preventing unwanted forces being applied directly to the patient.



## Wide option

100mm wider top option available.

Change the seat width from 650mm to 750mm wide to easily accommodate larger patients. Use our range of colour coding choices if required to assist in quick identification.



## Clear handset

Large simple functions with colour assistance.

Position the patient quickly and easily with the touch of a button - either by independently moving sections or applying one touch contouring.

## Specifications

**Size:** 780 x 1150mm overall (1750mm lying flat)

**Seat width:** 650mm with 100mm wider option available with 100mm thick premium, removable mattress

**Height Range:** 460 - 840mm with lie-flat and forward / reverse tilt

**SWL:** 300kg

**Castor Size:** Twin wheel 150mm castors with double steer

**Package Includes:** Standard IV, vertical oxy holder, dropdown footrest & meal tray

## Product Codes

Contour Recline: MPC-C-RET

Contour Recline Package MPC-C-RET-P

750mm wide top option: MPC-CA-WCT

# Contour Recline Endo

Enhance patient support with ease of access for physicians during procedures such as endoscopy.



## Key Benefits

**Safety & support:** The unique split dropside system allows staff the ability to drop one section without losing critical support for your patient and offers extra length.

**Space saver:** Fit more chairs in the same space and reduce patient transfers, time and OH&S risks.

**Hygiene conscious:** Smooth, gap free platform aids cleaning and helps prevent fluid ingress to meet the highest infection control standards.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance and fast guaranteed parts.

## Application

Ideal for Endoscopy / Colonoscopy

## Why Buy?

Make better use of valuable space and streamline your department with the Modsel procedure chairs. They are easy to move in and out of tight areas meaning a higher patient throughput and reduced waiting times. Patients can be positioned easily from sitting upright to a full recline, or lay completely flat with the stability of twin columns under the chair which can be used for CPR or specialised procedures. The unique twin dropside system\* ensures extra safety and efficiency for both the patient and carer. \*International design patents pending

# Contour Recline Endo



## Low and safe

Excellent 530mm low height with fold-under dropsides.

The ultra-low height is combined with dropsides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patients legs or clothes.



## Safer access

Lower the dropsides independently to ensure clear access to the patient without reducing side support.



## Clear handset

Large simple functions with colour assistance.

Position the patient quickly and easily with the touch of a button - either by independently moving sections or applying one touch contouring.



## Lies flat and tilts

The backrest and leg rest move independently or together from chair to bed with trendelenburg.

You now have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.



## Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors. The unique handle design correctly positions the carer while preventing unwanted forces being applied directly to the patient.



## Oxy holder options

Choice of backrest or dropside mount oxy bottle mounting.

## Specifications

**Size:** 850 x 1200mm overall (1850mm lying flat)

**Seat width:** Generous patient surface width of 750mm with 100mm thick premium, removable mattress

**Height Range:** 530 - 910mm with lie-flat and forward / reverse tilt

**SWL:** 300kg

**Castor Size:** Twin wheel 150mm castors with double steer

**Package Includes:** Standard IV, vertical oxy holder, drop down footrest, glide-away dropsides with no protrusions & meal tray

## Product Code

Contour Recline Endo: MPC-C-REN

Contour Recline Endo Package: MPC-C-REN-P

# Contour Recline E-Vertex

A world-first, fully electric procedure chair with touch control adjustable headrest for ease of operation, access and patient comfort.



## Key Benefits

**Full flexibility:** The infinitely adjustable headrest, range of chair heights with lie-flat and forward/reverse options places the surgeon and patient in exactly the right position

**Safety and support:** Increased leg clearance, with an adjustable, rigid locking headrest and wrist support accessory, brings patients close to protect surgeons from neck and back strain.

**Comfort and control:** Instantly increase patient comfort and enhance access with one-touch, electronic control of the scalloped or flat cushioned headrest; and pillow support accessory

**Hygiene conscious:** Smooth, gap free platform aids cleaning and helps prevent fluid ingress to meet the highest infection control standards.

## Application

Ideal for Eye, ENT, plastics and dental care.

## Why Buy?

The Contour E-Vertex medical procedure chair has been designed with direct feedback and insights from the country's leading Eye, ENT, plastics and dental professionals. The breakthrough design combines state-of-the-art technology and ergonomics to deliver exceptional functionality and support for procedures requiring close proximity, control for the practitioner and patient comfort. With convenient, touch-button handsets and user-friendly mechanisms, allowing a surgeon and their support team to quickly, safely and smoothly adjust the headrest, chair height and position for the procedure, patient transport and recovery.

[Also available as a Stretcher, view the Contour E-Vertex here.](#) \*International patents pending



# Contour Recline E-Vertex



## Low and safe

Excellent 460mm low height with fold-under dropsides.

The ultra-low height is combined with dropsides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patients legs or clothes.



## Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors.



## Effortless control

A clearly marked handset infinitely positions the headrest with the touch of a button.



## Lies flat


The backrest and leg rest move independently or together from chair to bed.

You now have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.




## Slim and clean

The ultra thin design safely hides all mechanical operation and has a one-piece, gap free top surface for easy cleaning.

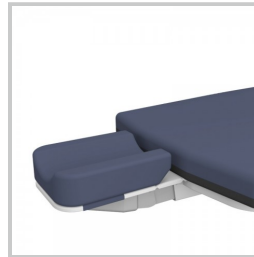

## Raise and lower

Adjust the height up and down while remaining level.

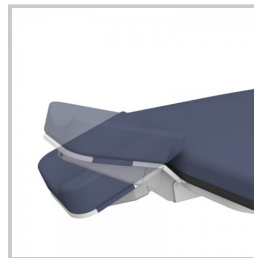

## Specifications

**Size:** 780 x 1150mm overall (1750mm lying flat)

**Seat width:** 650mm with 100mm wider option available with 100mm thick premium mattress

**Height Range:** 520 - 900mm with lie-flat and forward / reverse tilt

**SWL:** 300kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** standard IV, vertical oxy holder, drop down footrest, meal tray, standard scalloped or flat headrest cushion option

## Product Codes

Contour Recline E-Vertex: MPC-C-REVE-E

Contour Recline E-Vertex Package: MPC-C-REVE-E-P

Wrist Support: MPT-CA-EWS

Pillow Support: MPT-CA-CO-OSS

Flat Headrest Cushion Option: E0311

# Contour Recline EEG

A procedure chair designed to provide easy access for medical tests such as brain study.



## Key Benefits

**Create Space:** Quickly and easily remove the backrest section for clear head access.

**Hygiene conscious:** Smooth, gap free platform aids cleaning and helps prevent fluid ingress to meet the highest infection control standards.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with quality, care and customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance and fast guaranteed parts.

## Application

Ideal for Neurology or Epilepsy departments.

## Why Buy?

Make better use of valuable space and streamline your department with the Modsel procedure chairs. Patients can be positioned easily from sitting upright to a full recline, or lay completely flat with the stability of twin columns under the chair which can be used for CPR or specialised procedures. We've taken care with every aspect of design to ensure comfort and efficiency for both the patient and carer. Pop-up push handles ensure best operator positioning while preventing transfer movement to the patient. The Contour Recline EEG has a removable head section to provide complete clearance around the patients head.

# Contour Recline EEG



## Removable head section

Unlock and lift off the top section for clear access to the head.

The removal of the top cushion sections allows full access to the patient's head during EEG setup and testing.



## Low and safe

Excellent 460mm low height with fold-under dropsides.

The ultra-low height is combined with dropsides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patients legs or clothes.



## Access and comfort

The backrest and leg rest move independently or together from chair to bed.

Have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.



## Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors. The unique handle design correctly positions the carer while preventing unwanted forces being applied directly to the patient.

## Specifications

**Size:** 780 x 1150mm overall (1750mm lying flat).

**Seat width:** 650mm with 100mm wider option available with 100mm thick premium, removable mattress.

**Height Range:** 460 - 840mm with lie-flat and forward / reverse tilt.

**SWL:** 300kg.

**Castor Size:** Twin wheel 150mm castors with double steer.

## Product Code

MPC-C-REEG

# Contour Recline Barituff

The Barituff is a 500kg SWL bariatric chair bed with an extra-wide top and deeper seat.



## Key Benefits

**Low & safe:** The ultra-low height is combined with dropsides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patients legs or clothes.

**Purpose built:** Increased componentry strength in the core frame design operates with higher capacity columns to safely lift 500kg without overload.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Day Surgery, ED, Podiatry & General Ward

## Why Buy?

Having equipment that is robust enough, that you can completely rely on, is the greatest challenge when caring for bariatric patients.

We have increased componentry strength in the core frame design which operates with higher capacity columns to safely lift 500kg without overload. A wider top with a deeper seat has been standardized to comfortably cater for large patients. Heavy-duty, dark coloured vinyl is used on the mattress cushions to combat wear and staining. This stretcher chair has been built for bariatric patients.

# Contour Recline Barituff



## Low and safe

Excellent 480mm low height with fold-under drop sides.

The ultra-low height is combined with drop sides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patients legs or clothes.



## Easy egress

Forward tilt facilitates getting on and off.

For aged or disabled patients, a gentle incline and the long drop side armrests may assist in safer transfer.



## Lies flat

The backrest and leg rest move independently or together from chair to bed.

You now have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors. The unique handle design correctly positions the carer while preventing unwanted forces being applied directly to the patient.



## Large top design

The Barituff has a 200mm wider x 140mm deeper seat than the standard Recline.

The seat size is increased to 850mm wide x 600mm deep, easily accommodating larger patients. Use our range of colour coding choices if required to assist in quick identification.



## Clear handset

Large simple functions with colour assistance.

Position the patient quickly and easily with the touch of a button - either by independently moving sections or applying one touch contouring.

## Specifications

**Size:** 980mm wide x 1270mm upright (1750mm lying flat) overall

**Seat width:** 850mm wide x 600mm deep seat

**Mattress size:** 110mm pressure care mattress cushion with heavy duty cover

**Height Range:** 480 - 880mm with lie-flat and forward/reverse tilt

**SWL:** 500kg

**Castor size:** Twin wheel 150mm castors

**Package Includes:** Standard IV, vertical oxy holder, drop down footrest & meal tray

## Product Code

Contour Recline Barituff: MPC-C-REB

Contour Recline Barituff Package: MPC-C-REB-P

# Chair Oxy Holder

C size oxygen bottle holder which can be mounted to all four corners or the backrest of a chair.

## Key Benefits

- Low, high and backrest mount positions.
- Holds C-size oxygen bottle.
- Powder coated solid steel construction.

## Product Codes

- Vertical Oxy Corner Low Mount - MPC-CA-VO2
- Vertical Oxy Corner High Mount - MPC-CA-VUO2
- Vertical Oxy Backrest Mount - MPC-CA-BO2
- Vertical Oxy Dropside Mount - MPC-CA-DO2

## Why Buy

To cater for varied departmental requirements for specific oxy bottle positioning, we have developed a range of options. Some prefer fixed corner mount positions and others require a swinging style mounted from the backrest. There is also the option to support bottles vertically either above or below the patient surface.



Low Mount



High Mount



Backrest Mount



Dropside Mount



# Chair IV Pole

Height adjustable, non-rotational pole that fits all four corners of the chair and with under-top storage option.

## Key Benefits

- A strong and safe hook design
- Light and easy 'lift out' auto-lock hanger
- Non-rotational design to prevent tube tangling
- High strength locking for heavy loads
- Tough construction from stainless steel and nylon
- Easy 'pull-down' release from any angle
- Large 28mm diameter stand body for ease of pump mounting
- Friction retainer mount for effortless use
- Under top storage option available
- 705mm up to 1135mm total height
- 10kg weight capacity, 5kg per hook
- Only 1kg total weight

## Product Codes

- IV Pole; MPC-CA-IVP
- IV Storage Holder; MPC-CA-IVH

## Why Buy

Constructed tough from stainless steel and nylon, these clever non-rotational IV poles are designed to withstand the harshest use and make daily life easier by helping to prevent the tangling of critical soft tubing and cords. Simply lift out the hanger to any height and it automatically locks into an infinite position. Pull the lock body down from any direction to release and lower. This removable IV fits into all four corners of a chair top with the option to store underneath if there is provision.



Versatile



IV storage option

# Chair Foot Board

A removable board that locks into a padded foot rest or folds out for extra length as a leg support.

## Key Benefits


- Covered in heavy duty vinyl
- Fits all Recline leg supports
- Use as a foot rest or fold down to lengthen the leg support

## Product Code

- MPC-CA-DDF

## Why Buy

Clip in this foot board to provide a comfortable padded rest for the patients feet to help restrain them whilst moving the chair. It is covered in hard wearing vinyl that is easy to clean. The board can be folded down if the legrest is raised to provide extra length and support for the patient legs.

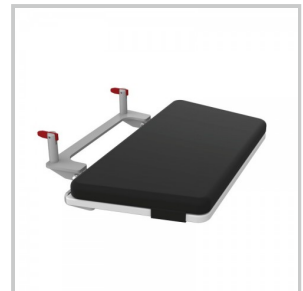

# Chair Meal Tray

Tough tray constructed from medical grade plastic, molded with cup and meal positions.

## Key Benefits


- Can be fitted to all four corners of the chair
- Swivels away for ease of access
- Folds down for convenient storage

## Product Code

- MPC-CA-MLT

## Why Buy

Quickly add this compact meal tray onto either side of the chair, lift and lock into position and swivel around to suit the patient. The tray folds down for easy storage and is constructed of a tough, one piece medical grade plastic that is very easy to clean. There are recesses to help prevent items from sliding off. This tray can be customised to any shape or size if required.

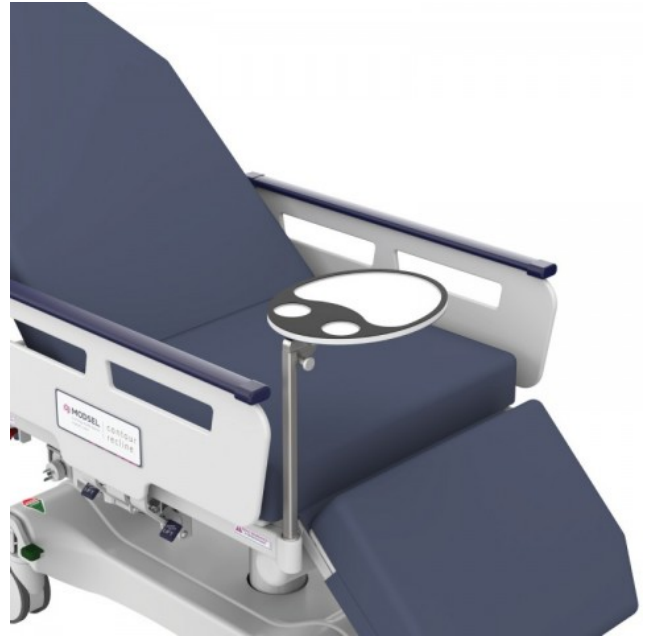

# Chair Arm Board

Lightweight armboard that clips onto the dropside rail and is padded for patients comfort.

## Key Benefits

- 25mm high density foam covered in PU material
- Solid medical grade plastic base board
- Clips on both inside and outside of the dropside
- Adjustable angles for best patient positioning

## Product Code

- MPC-CA-SAB

## Why Buy

Simply and securely lock the arm board in a choice of locations ranging from the front to the back of the dropside that suits the size and position of the patient. It may also be locked into a variety of angles to help the carer's access. The design is light to carry and store, quick to use and easy to clean.



# Chair Padded Armrests

Thick clip-on pads to soften the dropside armrests.

## Key Benefits

- High density foam core with slow recovery outer
- Covered in welded, pressure relieving stretch material
- 370mm long x 125mm wide x 210mm high
- 40mm of padding on top

## Product Code

- MPC-CA-PDA

## Why Buy

The Contour Recline Chair has been designed as a practical, clinical product that is functional and easy to clean. In some cases a patient may require extra support or comfort if spending longer periods of time in the chair. These easy to handle and store pads clip on over the dropside to widen and soften the armrest surfaces.



# Contour Energy

A transport chair with ZERO effort, 4-way power drive reducing the risk of strain during patient transport.



## Key Benefits

**Ergonomics & intuitive:** Drive effortlessly in any direction with only the slightest of touch, paired with perfect positioning of the drive controls for ease of use.

**Easy access:** The large footplates offer generous support to your patients and lift quickly for clear egress.

**Powerful movement:** A rotating electric drive wheel provides excellent traction, while center steering makes patient transport easier over longer distances.

## Why Buy?

Take all the human effort out of pushing and maneuvering chairs with an intuitive and intelligent drive system designed for easy use by any operator. Natural movement controls the speed and direction of the chair with the ability to stop very quickly by pulling back. Help prevent the most common carer strain injuries by taking the back twist out of moving heavy weight forwards, backwards and sideways. Patients are positioned comfortably with a gentle recline and feet fully supported. We've taken care with every aspect of design to ensure comfort and efficiency for both the patient and carer. Customise for a wide range of healthcare areas with relevant accessories.



# Contour Energy



## Ergonomic

Drive effortlessly in any direction by only light touch.



## Intuitive

Perfect positioning of the drive controls for the operator.



## Support

The large foot plates offer generous support to the patient.



## Easy access

The large foot plates lift quickly and easily for clear egress.



## Full clearance

Complete frontal clearance by lifting and rotating foot plates outwards.



## Removable armrests

The side armrests can be fully removed for multi direction egress.



## Versatile

Use the storage shelf during transport or customise with accessories such as IV poles, oxy holders and seat belt.

## Specifications

**Size:** 750mm W x 1300mm L x 1180mm H

**Seat Size:** 650mm W x 520mm D x 550mm H

**SWL:** 300kg

**Castor Size:** 150mm

**Package Includes:** standard IV & vertical oxy holder

## Product Code

Contour Energy: MPC-C-ENT-EDU

Contour Energy Package: MPC-C-ENT-EDU-P

# Contour Portare

An all-purpose electric transport and recovery stretcher with excellent low height.



## Key Benefits

**Safer solution:** The 400mm low platform height of this stretcher takes away the requirement to use step stools or unnecessary heavy lifting.

**Effortless control:** Sit and position your patient comfortably and then raise and lower them to the desired work height with the touch of a button.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Transfer stretcher suitable for most medical departments

## Why Buy?

Helping patients on and off a general purpose stretcher has never been this easy or safe with a design that allows the mattress base down to 400mm, lower than a standard office chair height! Transport must always be considered and with our new E-Point system, a strong grip with the floor and solid, perfectly centered steer wheel ensures the stretcher always accurately tracks and turns. We bring Modsel stretcher design into department areas such as emergency, theatre and day surgery helping to maximise the number of working units available.

# Contour Portare



## Ultra-low height

Very safe and easy for patients to get on and off.

The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## E-Point steering

A perfectly centred steer wheel ensures the stretcher always accurately tracks and turns.

Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction. Turning left or right from either end gives the same steering result.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 830mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.

## Specifications

**Overall Size:** 2135 x 815mm (965mm wide option)

**Patient Surface:** 1900 x 680mm (830mm wide option)

**Height Range:** 400 - 830mm

**SWL:** 300kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** 100mm premia mattress, folding IV, vertical oxy, foot endboard, drop down push handles, emergency CPR backrest release, 2nd handset & USB port

## Product Code

Contour Portare: MPT-C-PE2

Contour Portare Package: MPT-C-PE2-P

150mm Wider Top: MPT-CA-WCT

# Contour E-Vertex

A fully electric stretcher with touch control adjustable headrest for ease of use, access and superior stability.



## Key Benefits

**Full flexibility:** The infinitely adjustable headrest up and down tilt and excellent raise height allows the surgeon to place a patient in exactly the right position.

**Safety and support:** The compact, rigid locking headrest with increased leg clearance brings patients close and keeps them stable while minimising surgeon neck and back strain.

**Comfort and control:** Instantly increase patient comfort and enhance access with one-touch, electronic control of the scalloped or flat cushioned headrest; and lightweight removable frame with cushions to support a pillow during recovery.

**Slim and clean:** The ultra-thin design hides all mechanical operation and has a one-piece, gap-free top surface for ease of cleaning and infection control.

## Application

Ideal for Eye, ENT, plastics and dental care.

## Why Buy?

We know precision care requires 100% confidence in the equipment being used. That's why the Contour E-Vertex combines state-of-the-art technology and ergonomic design to deliver exceptional support for the performance of procedures requiring close patient proximity, control for the practitioner and patient comfort. The slimline design of this extremely stable, rigid locking headrest provides excellent leg clearance – bringing the patient closer. While the touch button electronic positioning system\* allows a surgeon or their support team to quickly, safely and precisely position a patient's head. [Also available as a procedure chair, see the Contour Recline E-Vertex here.](#) \*International patents pending.

# Contour E-Vertex



## Slim and clean

The ultra thin design safely hides all mechanical operation and has a one-piece, gap free top surface for easy cleaning.



## Effortless control

A clearly marked handset infinitely positions the headrest with the touch of a button.



## Raise and lower

Adjust the height up and down while remaining level.




## High tilt

Excellent raise height with up and down tilt adjustment.



## Low tilt

A practical low height with up and down tilt adjustment.

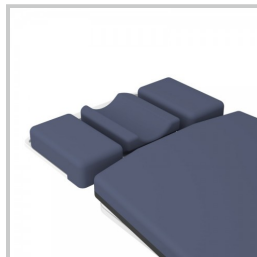

## Pillow Support option

Lightweight removable frame with cushions to help support a pillow during recovery



## Wrist Support option

Coated, 280mm wide, height adjustable and removable rest



## Fold-back design

The wrist support flips away to allow quick and easy access to the patient

## Specifications

**Overall Size:** 2200 x 815mm (965mm wide option).

**Patient Surface:** 2070 x 680mm (830mm wide option).

**Height Range:** 515mm-885mm.

**SWL:** 300kg.

**Castor Size:** 200mm castors and Ezi-Trak steering.

**Package Includes:** 75mm Premia Mattress, Vertical Oxy Holder (Rail Mount), Standard IV Pole, Foot Endrail, Pillow/Shoulder Support, 2nd Handset at Foot End of Stretcher & USB Port for charging of patient devices.

## Product Codes

Contour E-Vertex: MPT-C-VE-E

Contour E-Vertex Package: MPT-C-VE-E-P

Wrist Support: MPT-CA-EWS

Flat Headrest Cushion Option: E0311

# Contour Conceal

A purpose-built concealment stretcher for mortuary transport with superior functionality and customisable features.



## Key Benefits

**Careful concealment:** Fully customisable concealment stretcher with high quality, perfectly fitted vinyl cover in subtle grey or printed in one of three restful, full colour art designs.

**Effortless control:** Easy-drop flushfold sides and adjustable stretcher height ensures ease of transfer onto the Contour Conceal. Electric and hydraulic options available.

**Smooth and steady:** Robust, two-column design with 200mm castors and Ezi-Trak 5th wheel steering ensures stretcher stability and manoeuvrability in-transit

**Easy clean:** The Contour Conceal's smooth, one-piece platform and removeable, washable vinyl cover combine to make cleaning and hygiene control simple and effective.

## Application

Ideal for Wards, Emergency, Theatre, Intensive Care & Palliative

## Why Buy?

The Contour Conceal has been sensitively designed to assist you to provide patients who have passed away with a safe, secure, and dignified passage from wards to mortuaries. Offering full concealment, with your choice of subtle grey or calming rainforest, mountain or lakeside theme covers, the platform stretcher ensures each transfer is carried out with the utmost of ease, efficiency and respect. The twin column design is available as either an electric or hydraulic stretcher with a narrow or wide platform top. Both options are able to be adjusted to several height ranges and feature flushfold dropsides to facilitate the patient's smooth transfer onto the stretcher.

# Contour Conceal



Rainforest Theme



Lake Theme



Mountain Theme



Clear access

Lift off the heavy duty washable vinyl cover with full length access from both sides. Store the cover supports on convenient hooks.



Custom Options

Upgrade your Contour Conceal with orbit-drive, out door base or your choice of a grey or printed theme cover



Top width options

Narrow and wider top options available



Hydraulic option

Choose manual height adjustment if electric operation is not required



## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 700mm (590mm narrow & 850mm wide options)

**Electric Height Range:** 440-810mm (standard) up to 540-910mm

**Hydraulic Height Range:** 525-850mm

**SWL:** 300kg

**Castor Size:** 200mm castors & Ezi-Trak 5th wheel steering

**Options:** Power drive system and outdoor base upgrades available

## Product Code

Electric - MPT-C-CCE

Hydraulic - MPT-C-CCH

## Printed Cover Codes

MPT-CCA-LAKE

MPT-CCA-MOUNT

MPT-CCA-RAIN

# Contour Jazz

An Australian made transport and recovery stretcher that's a joy to use and designed for supreme patient safety and comfort.



## Key Benefits

**Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor

**Safety and support:** The ultra-low 400mm height makes it safer and easier for patients to get on and off – no need for step stools or unnecessary heavy lifting.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Bright and light:** Available in a choice of four fun and colourful themes to brighten their day (and yours).

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for: Emergency, Theatre & Day Surgery

## Why Buy?

Keeping patients safe, secure, and comfortable has never been easier with the Contour Jazz transport and recovery stretcher. With full, touch button electronic control and innovative E-Point steering system, the stretcher offers a smooth ride for patients and the kind of ease of operation and manoeuvrability that every caring professional craves. The Contour Jazz has been expertly designed to ensure you'll never miss a beat in even the most challenging environment or circumstances. Featuring a fast-action CPR backrest release, strong grip with the floor and an ultra-low height range that allows the mattress base down to 400mm. That's lower than a standard office chair height! Available in a range of colours and themes.

# Contour Jazz



Ready for a range of accessories



## Ultra-low height

Very safe and easy for patients to get on and off.

The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Music Theme



Sea Theme



Jungle Theme



Space Theme



## Fixed Control Panel

A fixed control panel on the foot end of the stretcher prevents unwanted misuse.

## Specifications

**Overall Size:** 2135 x 815mm (965mm wide option)

**Patient Surface:** 1900 x 680mm (830mm wide option)

**Height Range:** 400mm-830mm

**SWL:** 300kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** Handset and end controls, 100mm Premia mattress, emergency CPR back rest release, foot end board, drop down push handles, vertical oxy holder, folding IV, 16 riser dropsides and USB Port.

## Product Code

MPT-C-JAZZ

## Decal Codes

MPT-CJA-MUSIC

MPT-CJA-SEA

MPT-CJA-JUNGLE

MPT-CJA-SPACE

# Contour Classic

The most flexible custom design platform available for a transport and recovery stretcher.



## Key Benefits

**Customisable design:** Choose from a variety of dropside options and electric or hydraulic versions available.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Day Surgery, Recovery, Transit Lounge & Cardiac

## Why Buy?

The most flexible platform of all our stretcher range. The twin column design allows for several height range choices - we can build in a low height range for patient egress care or higher height ranges for better equipment or surgeon access with less strain during procedures.

Tailor make the Contour Classic with shorter, longer, narrower and wider top sizes and select from a range of dropside styles to suit specialised procedures or the patient type you are working with. To simply put it, there is a high level of customisation possible with many accessory mounting options and solutions.

# Contour Classic



## Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 810mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.



## Ultra-low height

Very safe and easy for patients to get on and off.

The 440mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.

## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)

**Electric Height Ranges:** 440 - 810 (std) up to 540 - 910mm

**Hydraulic Height Range:** 525 - 850mm

**SWL:** 300kg

**Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering

**Electric Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard, drop down push handles, 2nd handset & USB port

**Hydraulic Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard & drop down push handles

## Product Code

Electric: MPT-C-CE

Electric Package: MPT-C-CE-P

Hydraulic: MPT-C-CH

Hydraulic Package: MPT-C-CH-P

# Contour Exami

An economical, customisable, easy to move examination table available with or without dropsides.



## Key Benefits

**Safe & low:** The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting.

**Excellent height:** Electrically adjust the patient height up to 930mm helps to examine or treat your patient without having to stoop over, ensuring easy access and less stress.

**Comfortable:** 100mm fluid proof Protran mattress with a welded, pressure relieving cover.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for medical clinics, examination rooms, schools, prisons & mining sites

## Why Buy?

Helping patients on and off an examination table is easy and safe with a design that allows the mattress platform down to 400mm, lower than a standard office chair height! Uniquely, these stretcher examination tables are designed to be easily moved around and re-positioned due to their light-weight build and large castors. There are a range of accessories and customisations available to ensure the unit is designed to be priced and configured best to practically fit into your modern facility.

# Contour Exami



## Without dropsides

If you require a patient examination table, then this is worth considering. The standard Contour Exami without dropsides gives all the same functionality as an examination table, however has the additional value of colour choices and accessory mounting, while being very easy to move and reposition as required.



## Excellent high height

Don't strain your back, raise a patient up to your best working height. Lifting the platform height up to 830mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release option available.



## Customise it your way

Upgrades such as wider top, dropsides, base cover, colours and accessories. This is a very flexible platform with excellent customisation available. Change the standard Contour Exami from a basic examination table into an economy patient stretcher.



## Ultra-low height

Make it easier and safer for patients to get on and off. The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions. Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Go anywhere

Change from the standard 125mm twin to 175mm single castors for situations such as outdoor. If there is a requirement to wheel over rougher concrete, sand or grass terrains, this option gives larger diameter rubber wheels with 50mm extra ground clearance under the stretcher.

## Specifications

**Overall Size:** 2135 x 815mm (965mm wide option)

**Patient Surface:** 1900 x 680mm (830mm wide option)

**Height Range:** 400 to 830mm

**SWL:** 300kg

**Castor Size:** 125mm twin wheel castors, 4 brakes and 1 steer

**Includes:** 100mm Protran mattress, electric backrest that raises to 85 degrees, standard main power electric operation

**Options:** dropsides, base cover, battery back-up with cable retractor, larger castors and 6 endrail colours

## Product Code

Contour Exami (without dropsides); MPT-C-EXAMI

150mm wider top; MPT-CXA-WT

Dropsides; MPT-CXA-5RD

Base cover with storage; MPT-CXA-BC

Battery backup; MPT-CXA-BB

CPR backrest release; MPT-CXA-BQR

175mm large castor option; MPT-CXA-175



# Contour Barituff

A 500kg SWL extra-wide electric stretcher designed for Bariatric patients.



## Key Benefits

**Purpose built:** Increased componentry strength in the core frame design operates with higher capacity columns to safely lift 500kg without overload.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Day Surgery, ED, General Ward & Bariatric Care

## Why Buy?

Having equipment that is robust enough, that you can completely rely on, is the greatest challenge when caring for Bariatric patients. The Contour Barituff has increased componentry strength in the core frame design and operates with higher capacity columns to safely lift 500kg without overload.

The stronger and wider top comfortably caters for large patients, with a choice of mattress styles available to suit the type of departmental use. Stretcher stability is assured through an increased base size. Powered drive and weigh scales genuinely reduce manual handling risks to the carer. This stretcher has been built for Bariatric patients.

# Contour Barituff



## Ultra-low height

Easier for patients to get on and off.  
The 470mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions.  
Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 850mm (plus an extra 110mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.

## Specifications

**Overall Size:** 2135 x 1100mm

**Patient Surface:** 1900 x 980mm

**Mattress:** 110mm thick with 150mm thick option

**Height Range:** 470 - 850mm

**SWL:** 500kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** 110mm premia bari mattress, folding IV, vertical oxy, weigh scales, orbit drive, emergency CPR backrest release, 2nd handset & USB port

## Product Code

Contour Barituff: MPT-C-BE

Contour Barituff Package: MPT-C-BE-P

# Contour Deluxe

A transport stretcher featuring glide away siderails with zero transfer gap.



## Key Benefits

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Day Surgery, Recovery, Transit Lounge & Cardiac

## Why Buy?

Safely transfer patients with a zero transfer gap created by siderails that glide down under the top, leaving no obstruction on the side of the stretcher. Gas assistance ensures they are always light and easy for the operator and there is quick access with release catches on both ends.

Provide a greater level of support with these extra length siderails featuring horizontal bars and reduced gaps. Built for a lifetime, the fully welded and polished stainless steel construction leaves no risk of coatings to chip and rust, creating infection hazards or strength reduction.

# Contour Deluxe



## Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 940mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.



## Ultra-low height

Easier for patients to get on and off.

The 570mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.

## Specifications

**Overall Size:** 2135 x 815mm (965mm wide option)

**Patient Surface:** 1900 x 680mm (830mm wide option)

**Electric Height Range:** 570 - 940mm

**Hydraulic Height Range:** 630 - 960mm

**SWL:** 300kg

**Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering

**Includes:** Full length gas assisted siderails that are fully welded stainless steel, emergency CPR backrest release, X-ray subtop option

## Product Code

Electric - MPT-C-DE

Hydraulic - MPT-C-DH

# Contour Heli-Transfer

A stretcher designed to reduce helicopter patient transfer effort outdoors.



## Key Benefits

**Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for patient transfers

## Why Buy?

Safely and easily move a helicopter stretcher across onto this transfer stretcher, a platform designed to minimise effort whilst transporting patients over rougher outdoor surfaces. After benefiting from the smooth, flat bridge that lines up with the helicopter door, rapidly fasten the helicopter stretcher securely down with the four quick pull straps.

Reduce the trauma to patients and babies in cribs and the workload for carer's whilst transferring via the large diameter castors and very stable stretcher.

# Contour Heli-Transfer



## Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



## Orbit Drive Upgrade

Help prevent strain injury with this unique forward, back and sideways powered drive system.



## RACQ Life Flight

Built in backrest with a special thin high density mattress and split dropsides.



## Westpac

6 riser dopside and rubberised outdoor castor wheels.

## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 700mm (590mm narrow & 850mm wide options)

**Electric Height Ranges:** 550 - 920mm

**Hydraulic Height Ranges:** 530 - 860mm

**SWL:** 300kg

**Castor Size:** 200mm rubberised castors for outdoor shock absorption

**Package Includes:** tie down straps, orbit drive, all-terrain castors, flat stretcher top with tie down straps and gap bridge, fully customised top surface option to suit special purpose & comes standard without backrest

**Options:** Can be supplied with or without dropsides, emergency CPR backrest available

## Product Code

Electric: MPT-C-HE

Electric Package: MPT-C-HE-P

Hydraulic: MPT-C-HH

# Contour Portare-X

An ultra-low electric trauma stretcher with totally unobstructed x-ray clearance.



## Key Benefits

**Unobstructed x-rays:** Supported by just four outside corners, our unique x-ray system gives total freedom of movement for cassettes - even with an electric backrest!

**Safe & low:** The 430mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting.

**Smooth & steady:** Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for ED & Imaging

## Why Buy?

Helping patients on and off a trauma stretcher is safer and easier when the bed can be lowered down to 430mm, around the standard office chair height! This stretcher can feature our unique InfineX\* system which allows totally unobstructed head to toe clearance for effortless and efficient x-ray cassette positioning. Smoothly moving patients helps improve their experience and our E-Point steering delivers a strong grip to the floor with a solid, perfectly centered wheel to allow the stretcher to always accurately track and turn. The Modsel x-ray stretchers are designed to be a reliable, cost-effective performer, no matter what the shift brings.

\*Patent pending.

# Contour Portare-X



## Unobstructed x-ray tunnel

100% x-ray clearance of the entire body. Supported by just four outside corners, our unique patented InfineX system gives total freedom of movement for x-ray cassettes - even with an electric backrest! There are two styles available, another that allows the backrest to raise separately to the lower cassette support surface.



## Excellent high height

Quickly raise a patient up to the best working height. Lifting the platform height up to 880mm helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## E-Point steering

A perfectly centred steer wheel ensures the stretcher always accurately tracks and turns. Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction. Turning left or right from either end gives the same steering result.



## Cassette holder options

Holders can be produced with single or double sided handles and any depth or shaped rebates to suit all of the unique cassette sizes in use today



## Ultra-low height

Safer and easier for unwell patients to get on and off. The 430mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions. Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Easy customisation

Increase top width, change colours, mattress and dropside styles. Every trauma department has their own unique requirements to be able to function safely and efficiently.



## Specifications

**Overall Size:** 2135 x 815mm (965mm wide option)

**Patient Surface:** 1900 x 680mm (830mm wide option)

**Height Range:** 430 - 880mm

**SWL:** 300kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** 100mm premia mattress, folding IV, vertical oxy, foot endboard, emergency CPR backrest release, E-Point steering system, drop down push handles, 2nd handset & USB port

**Options:** Unique InfineX x-ray system option

## Product Codes

Contour Portare-X: MPT-C-PEX

Contour Portare-X Package: MPT-C-PEX-P

150mm Wider Top: MPT-CA-WCT

X-Ray Cassette Holder: MPT-CA-XCH-D

# Contour Multi-X

A trauma x-ray stretcher with ability to custom design and build to adapt to any emergency system.



## Key Benefits

**Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for ED / Imaging

## Why Buy?

This stretcher allows our unique InfineX\* tracking which gives totally unobstructed head to toe clearance for effortless and efficient x-ray positioning, with an electric backrest that can raise separately to the cassette support surface. We can help you get the most out of your expensive x-ray systems by building to suit the brand and equipment in use. Tailor make the Contour Multi-X with height range choices and shorter, longer, narrower or wider top sizes. Put safety first and select from a range of dropside styles to suit the department requirements.

\*Patent pending.

# Contour Multi-X



## Electric operation

Battery powered touch button control.  
Sit and position your patient comfortably and then raise them to the desired work height with ease.



## Hydraulic option

Non-powered, manually operated stretcher option.  
In remote facilities, or large departments, hydraulic stretchers may be more beneficial, with simple servicing and without the requirement of regular charging or power cords. Includes a tracking digital cassette carriage and holder.



## Electric standard

Standard powered stretcher in base configuration.  
Sit and position your patient comfortably and then raise them to the desired work height with ease.



## Ultra-low height

Easier for patients to get on and off.  
The 545mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Excellent high height

Preserve your back and raise a patient up to your best working height.  
Lifting the platform height up to 915mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Head and foot tilt

A generous 15 degrees top tilt both directions.  
Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.  
Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.

## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)

**Electric Height Range:** 545 - 915mm

**Hydraulic Height Range:** 605 - 930mm

**SWL:** 300kg

**Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering

**Electric Package Includes:** 100mm premia mattress, folding IV, dropdown footboard, vertical oxy, drop down push handles, 2nd handset & USB port

**Hydraulic Package Includes:** 100mm premia mattress, folding IV, dropdown footboard, vertical oxy & drop down push handles

## Product Code

Electric: MPT-C-XE-EB

Electric Package: MPT-C-XE-EB-P

Hydraulic: MPT-C-XH

Hydraulic Package: MPT-C-XH-P

X-Ray Cassette Holder: MPT-CA-XCH-D

# Contour Barituff-X

A 500kg SWL extra-wide electric trauma stretcher designed for Bariatric patients.



## Key Benefits

**Purpose built:** Increased componentry strength in the core frame design operates with higher capacity columns to safely lift 500kg without overload.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for ED / Imaging and Bariatric Care

## Why Buy?

Having equipment that is robust enough, that you can completely rely on, is the greatest challenge when caring for Bariatric patients. The Contour Barituff-X has increased componentry strength in the core frame design and operates with higher capacity columns to safely lift 500kg without overload.

The stronger and wider top comfortably caters for large patients, with a choice of mattress styles available to suit the type of departmental use. Stretcher stability is assured through an increased base size. Powered drive and weigh scales genuinely reduce manual handling risks to the carer.

This stretcher has been built for Bariatric patients.

# Contour Barituff-X



## Ultra-low height

Easier for patients to get on and off.  
The 600mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Head and foot tilt

A generous 15 degrees top tilt both directions.  
Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 980mm (plus an extra 110mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Cassette holder options

Holders can be produced with single or double sided handles and any depth or shaped rebates to suit all of the unique cassette sizes in use today

## Specifications

**Overall Size:** 2135 x 1100mm

**Patient Surface:** 1900 x 980mm

**Mattress:** 110mm thick with 150mm thick option

**Height Range:** 600 - 980mm

**SWL:** 500kg

**Castor Size:** 150mm twin wheel castors

**Package Includes:** 110mm Premia Bariatric mattress, folding IV, weigh scales, vertical oxy, orbit drive, 2nd handset & USB port

## Product Codes

Contour Barituff-X: MPT-C-BEX

Contour Barituff-X Package: MPT-C-BEX-P

X-Ray Cassette Holder: MPT-CA-XCH-D

# Contour Endosurge

An endoscopy imaging stretcher with a sliding top for quick and easy patient positioning.



## Key Benefits

**Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor.

**Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Endoscopy / Colonoscopy & Pain Management

## Why Buy?

Take away the effort of re-positioning a heavy C-Arm X-Ray camera. For quick and easy movement of the patient, unlock the stretcher top and slide it a generous distance back or forwards.

Our aim is to provide more freedom and take strain off your backs. The cutaway base cover allows more clearance for the camera under the stretcher whilst bringing it closer to the floor for a lower working height.

# Contour Endosurge



## Slide to head

Unlock and move the top 140mm towards the head end from centre position.

Release the top lock and easily slide the stretcher top back or forwards to position your patient without having to significantly move the heavy C-arm imaging camera.



## Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



## Slide to foot

Unlock and move the top 260mm towards the foot end from the centre position.

Release the top lock and easily slide the stretcher top back or forwards to position your patient without having to significantly move the heavy C-arm imaging camera.

## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)

**Electric Height Ranges:** 570 - 940mm

**Hydraulic Height Ranges:** 630 - 960mm

**SWL:** 300kg

**Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering

**Sliding Top:** 400mm slide; 140mm to head and 260mm to foot from centre

**Electric Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard, drop down push handles, emergency CPR backrest release, split dropsides, 2nd handset & USB port

**Hydraulic Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard, drop down push handles & split dropsides

**Options:** Split dropside option (shown) and mid-position catch upgrades

## Product Code

Electric: MPT-C-EE

Electric Package: MPT-C-EE-P

Hydraulic: MPT-C-EH

Hydraulic Package: MPT-C-EH-P



# Contour Endo-X

The endoscopy imaging stretcher with a 'C' base for complete freedom of camera movement.



## Key Benefits

**Optimal design:** Increased column spacing with left or right hand chassis

**Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.

**Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.

**Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

## Application

Ideal for Endoscopy & Imaging

## Why Buy?

An increased distance between the lifting columns of this stretcher allows excellent coverage with the C-Arm X-Ray camera, avoiding the risks of re-positioning and moving patients during procedures. Complete freedom of movement is also facilitated by the single rear bar joining the stretcher base, which creates ample space for the camera and leaves no restriction to the working height required by staff.

# Contour Endo-X



Base option



## Ultra-low height

Very safe and easy for patients to get on and off.

The 570mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



## Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 940mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



## Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



## Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.

## Specifications

**Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)

**Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)

**Height Range:** 570 - 940mm

**SWL:** 300kg

**Castor Size:** 200mm large bearing castors

**Package Includes:** 100mm premia mattress, folding IV, dropdown footboard, vertical oxy, split dropside, emergency CPR backrest release, C-shaped base for maximum camera clearance, 2nd handset & USB port

**Options:** Split dropside option (shown) and mid-position catch upgrades

## Product Codes

Contour Endo-X: MPT-C-EX

Contour Endo-X Package: MPT-C-EX-P

Split fold dropside option: MPT-CA-SDS

# Stretcher Mattress

High grade mattress with excellent durability and pressure relieving comfort for patient stretchers.

## Key Benefits

- Thicknesses that aid egress and imaging
- Protran mattress - single foam layer
- Premia mattress - 3 effective foam layers
- Premia Barituff mattress - 4 support layers (110mm thick 60 - 350kg, 150mm thick 45 - 400kg therapeutic support loads)
- Welded 'double coated' stretch cover
- Non-slip base design
- Permanent care label
- New technology fluid proof zip (excludes 75mm mattress)
- MADE IN AUSTRALIA

## Product Code

- 100mm Protran Mattress; MPT-CM-100-PTP
- 100mm Premia Mattress; MPT-CM-100-PRP
- 110mm Barituff Mattress; MPT-CM-110-PRB
- 150mm Barituff Mattress; MPT-CM-150-PRB

## Mattress Size



- Standard; 680 W x 1900mm L
- Narrow; 570mm W x 1900mm L
- Wide; 830mm W x 1900mm L
- Barituff; 980mm W x 1900mm L

## Mattress Shape

- Square Ends (electric stretcher)
- Truncated End (hydraulic stretcher)

## Why Buy

The Contour Premia mattress has been designed to optimize patient weight capacity and comfort by using a selection of quality foam layers to help in both preventing patients from bottoming and pressure reduction. The Contour Protran mattress is recommended for short stay patients with a single medium density foam section to facilitate special patient transfers or clear imaging.

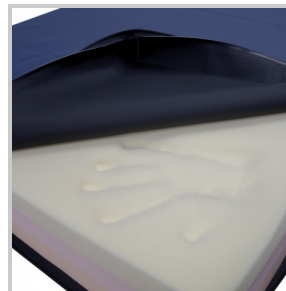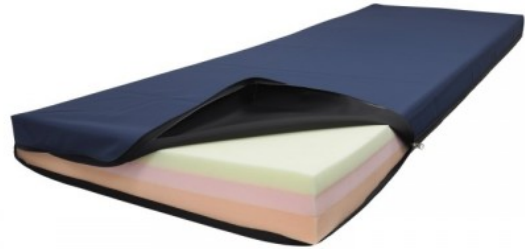

Pressure care design




Welded cover




Sealed zip



Care label

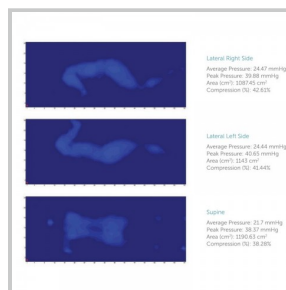


Pressure mapped

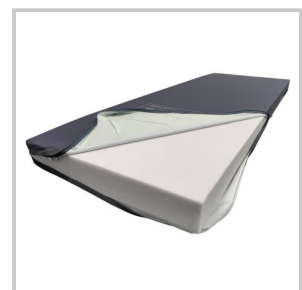

Protran

# Premium Stretcher Mattress

High performance mattress with outstanding durability and patient support within demanding acute environments.

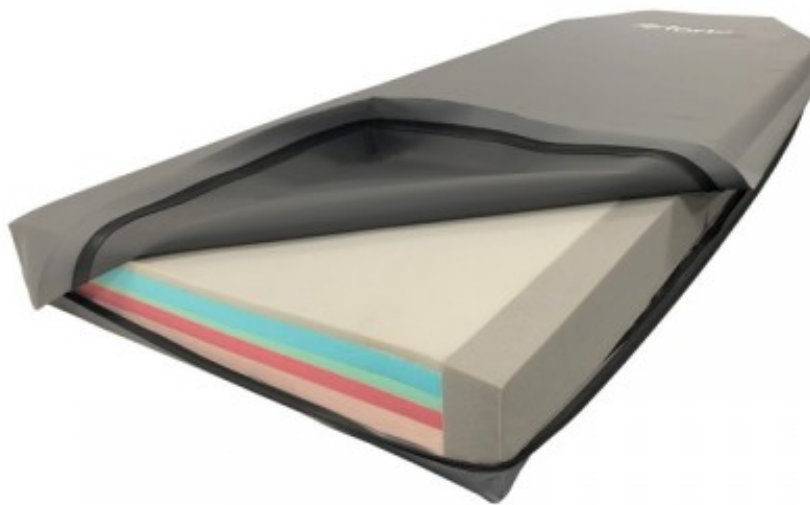

## Key Benefits

- **Crucial support:** 5-layered high performance core for demanding environments.
- **Structural stability:** Integrated profile stabilizing sides and unique hinging system.
- **Fluid control:** Double coated, welded polyurethane cover with fluid proof zip.
- **Body distribution:** Superior 110mm mattress thickness with up to 350kg mass range.
- **Peace of mind:** Australian made plus offering a 5 year core foam and 4 year cover warranty.

# Premium Stretcher Mattress



## Stabilizing sides

High density bars down each side help to prevent roll off and assist the patient during egress by increased side strength of the mattress.




## Premium pressure relief

The special combination of foam densities and layers provide superior patient support.



## Profile cut sides

Strategically placed reliefs allow this thicker and denser mattress to conform to the stretcher surface contouring.

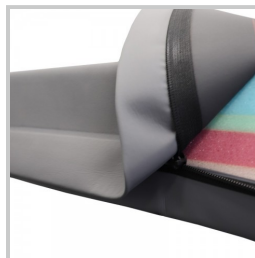

## Fluid proof zip

The unique zip design seals when closed to resist fluid penetration, with the added protection of a premium waterfall flap.



## Non-slip base

A specially coated base fabric helps to prevent unwanted movement of the mattress and grip the stretcher surface without requiring mechanical fastenings. This allows quick access and fast cleaning of the patient surfaces.

## Why Buy

The **Premia Forte** pressure relieving radiolucent support surface is our high spec version of the standard Contour Premia mattress. The specialised core structure provides best protection to patients at risk of injury without compromising support in crucial areas. A Premiflex stretcher cover provides ultimate patient comfort and protection. This mattress delivers an effective outcome - combining both high risk pressure care and longevity for demanding stretcher applications.

## Product Code

- Truncated End (hydraulic stretcher); MPT-CM-110-PRF
- Square Ends (electric stretcher); MPT-CM-110-PRPF

## Mattress Size

- Standard: 680 W x 1900mm L
- Narrow: 570mm W x 1900mm L
- Wide: 830mm W x 1900mm L

# Orbit-Drive

Taking patient transport to a new level in carer welfare with zero effort, 4-way power drive.



## Key Benefits

- Forward, back and sideways powered drive
- Intuitive height adjustable controls with emergency stop
- Powerful electric drive centre steer wheel

# Orbit-Drive



## Adjustable controls

Lower the drive controls down below the mattress height for enhanced patient access and specialised accessory use.



## Adaptable

Install the power drive system on many of the Contour stretcher models



## Powerful drive

An electric driven wheel pressured onto the floor that provides excellent traction and center steering.



## Sideways movement

The unique rotating design allows the stretcher to be driven in any direction - forwards, backwards and sideways!

## Why Buy

Take all the human effort out of pushing and jostling stretchers with an intuitive and intelligent drive system designed for easy use by any operator. Natural movement controls the speed and direction of the stretcher with the ability to stop very quickly by pulling back.

Help prevent the most common carer strain injuries by taking the back twist out of moving a stretcher forwards, backwards and sideways. Importantly, we haven't overlooked safety features such as emergency stop and system lock out, which are standard. Add this system to our traditional vertical column stretcher designs such as Contour Barituff, Classic, Deluxe and Multi-X to improve both standard and Bariatric patient transport.

## Product Code

- MPT-CA-EDU

# Dropside Options

There are a range of dropside options available to customise any stretcher to suit your special requirements.



## Key Benefits

- Multi-riser dropside options for closer spacings
- Split fold option for safer patient access
- Horizontal rail designs for longitudinal support
- Flush folding option for mattress transfer or x-ray
- Mid position lock option for easier patient access



# Dropside Options



## 16 riser dropsides

Close up the spacing's to 100mm centers with extra risers for a safe side rail for patients requiring extra support.

Choice of folding from head end towards foot or foot end towards head (pictured).



## 7 riser dropsides

Close up the spacing's to 250mm centers with extra risers for a safe side rail for patients requiring extra support.

Choice of folding from head end towards foot (pictured) or foot end towards head.



## Split folding dropsides

Enhance access to either the head or foot end of the patient without compromising safety.

Fold all four sections down together or independently. Supplied standard with mid-position catches.



## Flush folding dropsides

Fold down the dropsides to flush with trolley top, allowing slide transfer of mattress or horizontal x-ray.

Uses the slim aluminium handrail section. Choice of folding from head end towards foot (pictured) or foot end towards head on any dropside riser configuration.



## Mid-position lock

Pair of catches with a mid locking position for special access to the patient.

Choice of folding from head end towards foot (pictured) or foot end towards head. Supplied standard on split dropsides.



## Deluxe dropsides

Full length glide-away siderails that provide horizontal support and a zero transfer gap.

## Why Buy

We understand that there are different ways that patients must be transported and procedures performed. Safety comes first, which is why we've developed a selection of side rail options, to ensure you can have the right outcome for your department's requirements.

## Product Codes

- 16 riser dropsides - MPT-CA-16RD
- 7 riser dropsides - MPT-CA-7RD
- Split fold dropsides - MPT-CA-SDS
- Flush fold dropsides - MPT-CA-FFD
- Mid position lock - MPT-CA-CMP
- Deluxe dropsides - MPT-C-D

# Stretcher IV Poles

Include a standard removable or fold down IV Pole with your stretcher.

## Key Benefits

- A strong and safe hook design
- Light and easy 'lift out' auto-lock hanger
- Non-rotational design to prevent tube tangling
- High strength locking for heavy loads
- Tough construction from stainless steel and nylon
- Easy 'pull-down' release from any angle
- Large 28mm diameter stand body for ease of pump mounting
- Friction retainer mount for effortless use
- Removable and permanent fold down options
- 705mm up to 1135mm total height
- 10kg weight capacity, 5kg per hook
- Only 1kg total weight

## Product Code

- Standard IV Pole - MPT-CA-IVP
- Folding IV Pole - MPT-CA-FIV

## Why Buy

Constructed tough from stainless steel and nylon, these clever non-rotational IV poles are designed to withstand the harshest use and make daily life easier by helping to prevent the tangling of critical soft tubing and cords. Simply lift out the hanger to any height and it automatically locks into an infinite position. Pull the lock body down from any direction to release and lower. The standard removable IV fits into all four corners of a stretcher top and stores underneath in the base if there is provision. The folding IV fits onto either end of the stretcher top and can be installed to fold down either to the left or right.



Standard IV storage



Folding IV Pole

# Oxy Holders

A selection of coated stainless steel oxy bottle holders that can be mounted vertically or horizontally.

## Key Benefits

- Tough, powder coated stainless steel basket construction
- Vertical basket size for up to 120mm diameter bottle; 320mm deep x 150mm wide
- A range of vertical mount positions - low, high with swivel or 45 and 90 degree fixed angles
- Horizontal basket size for up to 140mm diameter bottle; 600mm long x 180mm wide
- Horizontal basket can vary in hanging height depending on stretcher models; typically 280mm

## Product Code

- Vertical Rail Mount - MPT-CA-VO2
- Vertical Corner Mount Swivel - MPT-CA-VCO2
- Vertical Corner Mount Fixed 45 - MPT-CA-VCO2-45
- Vertical Corner Mount Fixed 90 - MPT-CA-VCO2-90
- Vertical Corner High Mount Swivel - MPT-CA-VCH-S
- Vertical Corner High Mount Fixed 45 - MPT-CA-VCH-45
- Vertical Corner High Mount Fixed 90 - MPT-CA-VCH-90
- Horizontal Oxy Holder - MPT-CA-HO2

## Why Buy

Due to so many wide and varied departmental requirements for specific oxy bottle sizes and positioning, we have developed a range of options. Larger bottles can be laid horizontally to ensure they are kept out of the way and smaller bottles are commonly mounted vertically. Some prefer 'fixed', non-moving positions and others require 'swinging' options that allow the bottles to be moved aside if they become an obstruction during transport. There is also the option to support bottles vertically both above and below the patient surface.



Vertical Corner Mount



Vertical Corner High Mount



Horizontal Oxy Holder



Horizontal Oxy Holder

# Dropdown Push Handles

Strong fold-away handles with rubber grip for pushing the trolley.

## Key Benefits

- Strong stainless steel construction
- Lift up and fold down
- Tough hygienic hand grips
- Tall head end handle option - 400mm high x 600mm wide (350mm between handles)
- Foot end handle option - 160mm high x 600mm wide (350mm between handles)

## Product Codes

- Tall head end handles; MPT-CA-TDPH
- Foot end handles; PT-CA-DPH

## Why Buy

To allow space for the many removable and common accessories used at the foot end of the stretcher, the handles are shorter to fold completely away with no obstruction to the accessory mounting positions. There is an option for a tall version of these handles to be mounted at the head end of the stretcher which provide high pushing points and ample clearance from the moving backrest.



Head end handles folded



Foot end handles



Foot end handles folded

# Document Holder

Strong powder coated wire basket that is hung or mounted for document storage.

## Key Benefits

- Ultra-tough, powder coated, 5mm steel wire construction
- External size; 390mm wide x 80mm deep x 230mm high
- Internal size; 380mm wide x 60mm deep x 225mm high
- Hang on the inside or outside of a stretcher end rail
- Wall and accessory mounting options

## Product Code

- MPT-CA-DCH

## Why Buy

Document holders can be used in a variety of ways for many different storage requirements. In the past we have mounted holders on either end or in special places around a stretcher, off accessories such as the dropdown foot board and even on the walls around the stretcher holding bays. We can customise sizes and designs and will work with you to ensure you can maximise department efficiency.



Removable holder

# End Rail

An end rail that can help to retain the mattress, patient or be used as a push handle.

## Key Benefits

- Strong, 25mm diameter coated stainless steel rail
- 600mm wide x 200mm high
- Fits into foot end or optional head end accessory sockets

## Product Codes

- End rail; MPT-CA-FER
- Head end accessory sockets; MPT-CA-ASH (pair)

## Why Buy

Quickly drop this rail into the accessory sockets (optional accessory on the head end of a stretcher, excludes Contour Vertex) and use it as a higher strong point to move the stretcher. This rail has a variety of additional uses such retaining a mattress or pillow from sliding off or a bar for a patients feet to push against. These rails can be used on either end of the stretcher or both ends together.



Head end option



Both ends

# End Board

An end board that can help to retain the mattress, patient or be used as a higher pushing position.

## Key Benefits

- Strong, 25mm diameter powder coated steel rail
- Tough, hospital grade solid plastic infill
- 600mm wide x 345mm high
- Fits into foot end or optional head end accessory sockets

## Product Codes

- End board; MPT-CA-FRTI
- Head end accessory sockets; MPT-CA-ASH (pair)

## Why Buy

Quickly drop this board into the accessory sockets (optional accessory on the head end of a stretcher, excludes Contour Vertex) and use it as a higher strong point to move the stretcher. This end board has a variety of additional uses such as retaining a mattress or pillow from sliding off or a flat surface for a patients feet to push against. These boards can be used on either end of the stretcher or both ends together.



Head end option



Both ends



# Dropdown Foot Board

Padded fold-down foot board to increase the length of the patient surface.

## Key Benefits

- High density foam covered in welded stretch material
- Tough board construction in a one-piece hospital grade plastic
- Stainless steel lift and lock folding hinge
- 530mm wide x 330mm high (250mm high cushion)
- Adds an extra 310mm to the length of a mattress

## Product Code

- MPT-CA-DDF

## Why Buy

Padded for patient comfort, the removable dropdown foot board has molded handles for pushing the stretcher and with a simple 'lift and lay' action, folds down to quickly increase the length of the mattress area.



Fold down



# Folding Monitor Board

A strong frame with push handles and folding board with an adjustable tie strap for retaining a monitor.

## Key Benefits

- Powder coated steel frame
- Cross bars for patient feet protection
- One-piece hospital grade plastic board
- 50mm wide adjustable tie strap
- 450mm wide x 400mm deep tray surface
- 600 wide x 470mm high from mattress base
- 15kg weight capacity

## Product Code

- MPT-CA-FMB

## Why Buy

Add this removable monitor board accessory to the foot end of a stretcher to provide a solid push handle and fold-away board for securely mounting a monitor. The board hangs vertically out of the way and can be easily rotated outward and over to a stable, internal, horizontal rest position.



Fold over to a table



Writing surface option



Writing surface extended

# Folding Monitor-Foot Board

A strong frame with both a folding foot and monitor board with an adjustable tie strap for retaining a monitor.

## Key Benefits

- Powder coated steel frame
- One-piece hospital grade plastic board construction
- 600 wide x 410 deep x 540mm high overall size
- 520mm wide x 380mm deep monitor tray
- Monitor tray is 540mm high from mattress base
- 15kg weight capacity
- 50mm wide adjustable tie strap for monitors
- 500 wide x 250mm high foot board
- Adds 320mm to the length of the mattress

## Product Code

- MPT-CA-FMFB

## Why Buy

Add this removable monitor board accessory to the foot end of a stretcher to provide a solid push handle and fold-away board for securely mounting a monitor. The board hangs vertically out of the way and can be easily rotated outward and over to a stable, internal, horizontal rest position. Padded for patient comfort, the dropdown foot board folds with a simple 'lift and lay' action to quickly increase the length of the mattress area.



Fold over to a table



Fold down foot board



Writing surface option



Writing surface extended

# Weigh Scales

Add a weigh scale system to Contour Stretchers for ease of patient measuring and tracking.

## Key Benefits

200g accuracy Auto off with weight memory function Freeze weight to allow for additional accessories Excludes Contour Portare stretcher models

## Product Code

- MPT-CA-WS



Easy to use

# Stretcher Colour Options

A selection of colours to customise your Modsel stretchers for unique identification.

## Key Benefits

- Tough extruded covers for a comfortable hand grip and bump protection.
- Used on stretcher end rails and dropside hand rails.
- Easy to replace for colour change or wear and tear.

## Product Code

- Midnight Blue - MPT-CC-MB
- Flame Red - MPT-CC-FR
- Deep Violet (Purple) - MPT-CC-DV

## Why Buy

Customise your stretcher end rail and dropside colours to suit your facility requirements. Our colour trims have been used in many ways such as departmental zoning between emergency, theatre and outpatients, highlighting special units such as wide top, x-ray, procedure, Bariatric or simply brightening up a children's ward.



Midnight Blue



Flame Red



Deep Violet (Purple)

# Dropside Padding

Provide extra protection for recovering patients with a selection of padded bumper systems for stretcher drop-sides.



## Key Benefits

- Stiff board backed bumper system
- Lift and roll feature allowing use without removal
- Quick install pocket system
- 25mm high density padding
- Welded cover materials for easy cleaning
- Sizes to fit any dropside style

# Dropside Padding



## Padded Bumpers

Stiff plastic backed bumpers with welded stretch padding that resists high limb force.



## Unique access

Lift and roll to the outside of the dropside, allowing raise and lowering without removal.

Wall hanging point incorporated for easy storage.



## Padded Pockets

Tough vinyl padding on both sides of the dropsides protecting both the patient and carer.



## Removable design

Unclip and quickly lift off for unrestricted patient access and easy cleaning.



## Sizing options

Both bumpers and pockets can be supplied as full length versions or to fit different dropside styles.

## Why Buy

A recovering or restless patient can cause themselves serious harm on the metal risers and rails of standard dropsides. We have developed some ways for the carer to quickly and easily increase the level of protection and provide a soft, padded surface for patients limbs. Our board system can be flipped to the outside of the stretcher allowing the dropsides to be raised and lowered without having to remove the bumpers from the trolley. Pockets can slide on over the dropside which are quick to use, however must be removed to lower the sides.

## Product Codes

- Padded Bumpers (pair); MPT-CA-PDB-P
- Padded Pockets (pair); MPT-CA-PDB-D

# Meal Tray

Tough tray constructed from fluid proof board with a patient comfort cutaway.

## Key Benefits

- Clips onto the stretcher dropside rail
- Board sizes to suite narrow, standard and wide stretcher tops
- Very tough, easy to clean fluid proof design

## Product Code

- MPT-CA-MLT

## Why Buy

Quickly add this compact meal tray which can be supplied to clip onto the stretcher dropside. The tray is constructed of a tough, one piece medical grade board that is very easy to clean. There is a recess to ensure patient comfort. This tray can be customised to any shape or size if required.



# Aquatuff Electric

A stainless steel shower trolley with electrically operated 300kg SWL, wide top and backrest options.



## Key Benefits

**Safe & secure:** The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfers.

**Hygiene conscious:** Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.

**Custom adjustments:** Raise or lower the height of your trolley to suit bed transfer or best working positions. You can also lock the removable end rails into two positions, allowing 120mm extra space for longer patients

## Application

Designed for heavy use and demanding environments.

## Why Buy?

Having equipment that is robust and completely reliable is of the utmost importance when caring for larger or heavier patients. We have over-engineered the componentry strength in this design and our decision to use total stainless steel reinforces the lifetime of value that this bariatric shower trolley will give. The electric operation helps to safely lift 300kg without overload, whilst four twin wheels with brakes, electric backrest and wider top options all increase versatility in special needs.



# Aquatuff Electric



## Standard design

The standard top has a full length dropside with an electric backrest raise option for extra comfort if required.



## Safe transfer

The dropside folds underneath the top and the liner side lays down to provide gap free patient transfer.



## Easy drainage

Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.



## Height adjustment

Raise or lower the height of your trolley to suit bed transfers or best working positions.



## Length adjustment

Lock the removable end rails (on each end of the top) into two positions, allowing 120mm extra space for longer patients.



## Wide top option

Add a 100mm wider top for larger patients with wide shoulders.

## Specifications

**Size:** 795 W x 1950mm including bumpers

**Liner Inner Base Size:** 550 W x 1700 L x 180mm Deep

**Standard Hose Length:** 1100mm

**SWL:** 300kg

**Height Range:** 680 to 900mm

**Castor Size:** Quality 125mm twin wheel castors

**Construction:** Powder coated stainless steel

**Junior Size:** 795 W x 1680mm

**Junior Liner Inner Base Size:** 560 W x 1450 L x 180mm Deep

**Wider Comfort Option:** 100mm

## Product Code

Aquatuff Electric; MST-AQTE

Aquatuff Electric Junior; MST-AQTE-J

Electric backrest option (excludes Junior size) - MST-SA-EBA

100mm wider top option (excludes Junior size) - MST-SA-EWT

# Change Table

A purpose built, non-slip change mat on a sturdy, height adjustable and safe patient platform.



## Key Benefits

**Safe & secure:** The fold up dropsides and removable end rails ensure high levels of patient safety when required.

**Customisable design:** Tailor to your requirements with options such as electric high/low, 300kg SWL and wide tops.

## Why Buy?

This change mat expands the use of the Aquarius trolley into a very safe, height adjustable change table with multi-position, removable end rails and full length dropsides that fold back underneath the trolley top for ample work clearance. The mat securely fastens underneath the trolley top to prevent sliding and movement. This trolley is designed for harsh use and able to be rigorously cleaned.

# Change Table



## Safe design

The fold up dropsides and removable end rails ensure a high level of patient safety when required.



## Customise

Tailor to your requirements with options such as electric high/low, 300kg SWL and wide tops.

## Specifications

**Size:** 795 W x 1950mm including bumpers and 25mm high density foam core

**Patient Surface Size:** 600 W x 1820mm Long

**Height Range:** 60 to 900mm (depending on trolley model)

**SWL:** Hydraulic 180kg or electric 300kg

**Castor Size:** Quality 125mm twin wheel castors with total lock and steer

**Options:** Both junior and wider sized available

## Product Code

MST-SA-CTU

