



Shower Bathing Trolleys



Shower Bathing Trolleys

Our mobile shower trolleys are constructed for both patient safety and ease of use by allied health professionals, and feature a clean design with durable materials that are ideal for harsh environments.

Leading Supplier of Shower Trolleys in Australia

Perfect for use as a shower bed or stretcher, these designs allow for a stable, reliable solution for easy bathing across the board. Some options in our range include designs specially made for bariatric patients, while others offer complete privacy features.



Aquarius



Aquatuff



Aquatuff Bari



Aquatuff Electric



Change Table



Custom Liner Colours

Aquarius

A standard shower trolley designed to meet budget constraints.



Key Benefits

Safe & secure: The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfers.

Hygiene conscious: Tilt the top 6 degrees and lower the drain hose tip to quickly remove waste water.

Custom adjustments: Raise or lower the height of your trolley to suit bed transfer or best working positions. You can also lock the removable end rails into two positions, allowing 120mm extra space for longer patients.

Application

Designed for standard bathroom use.

Why Buy?

We are able to bring our Modsel shower trolley design quality to you at an affordable price, helping to meet tight budget constraints and maximising the number of working units available.

The refined framework and proven mechanics awards long life value and consists of considered, easy to clean surfaces coupled with quality twin wheel castors to ensure the highest level of maneuverability.

Aquarius



Safe transfer

The drop sides fold underneath the top and the liner side lays down to provide gap free patient transfer.



Easy drainage

Tilt the top 6 degrees and lower the drain hose tip to quickly remove waste water.



Height adjustment

Raise or lower the height of your trolley to suit bed transfers or best working positions.



Length adjustment

Lock the removable end rails (on each end of the top) into two positions, allowing 120mm extra space for longer patients.



Junior size option

A special junior sized shower trolley developed for small patients. Working with children or small adults may not require a full sized shower trolley. Our junior sized version is also easier to move around in tight facilities and for storage.

Specifications

Size: 795 W x 1950mm including bumpers

Liner Inner Base Size: 550 W x 1700 L x 180mm Deep

Standard Hose Length: 1100mm, custom lengths available

SWL: 180kg

Height Range: 680 to 880mm

Castor Size: Quality 125mm twin wheel castors with total lock and steer

Construction: Powder coated steel

Junior Option: 795 W x 1680mm including bumpers

Junior Liner Base Size: 550 W x 1450 L x 180mm Deep

Product Codes

Aquarius - MST-AQM

Aquarius Junior - MST-AQM-J

Aquatuff

A stainless steel shower trolley with hydraulically operated 300kg SWL and wide top option.



Key Benefits

Safe & secure: The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfers.

Hygiene conscious: Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.

Custom adjustments: Raise or lower the height of your trolley to suit bed transfer or best working positions. You can also lock the removable end rails into two positions, allowing 120mm extra space for longer patients.

Application

Designed for heavy use and demanding environments.

Why Buy?

Having equipment that is robust and completely reliable is of the utmost importance when caring for larger or heavier patients. We have over-engineered the componentry strength in this design and our decision to use total stainless steel reinforces the lifetime of value that this bariatric shower trolley will give. The hydraulic operation helps to safely lift 300kg without overload, whilst four twin wheels with brakes and wider top all increase versatility in special needs.

Aquatuff



Safe transfer

The drop sides fold underneath the top and the liner side lays down to provide gap free patient transfer.



Easy drainage

Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.



Height adjustment

Raise or lower the height of your trolley to suit bed transfers or best working positions.



Length adjustment

Lock the removable end rails (on each end of the top) into two positions, allowing 120mm extra space for longer patients.



Wide top option

Add a 100mm wider top for larger patients with wide shoulders.

Specifications

Size: 795 W x 1950mm including bumpers

Liner Inner Base Size: 550 W x 1700 L x 180mm Deep

Standard Hose Length: 1100mm, custom lengths available

SWL: 300kg (electric model available)

Height Range: 660 to 900mm

Castor Size: Quality 125mm twin wheel castors with total lock and steer

Construction: Powder coated stainless steel

Junior Option: 795 W x 1680mm including bumpers

Junior Liner Inner Size: 550 W x 1450 L x 180mm Deep

Wider Comfort Option: 100mm (internal min width of 660mm)

Product Code

Aquatuff Hydraulic; MST-AQT

Aquatuff Hydraulic Junior; MST-AQT-J

100mm wider top option - MST-SA-EWT

Aquatuff Bari

A robust, fully electric 300kg SWL shower trolley with extra wide base and top for Bariatric patients.



Key Benefits

Safe & secure: The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfers.

Hygiene conscious: Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.

Custom adjustments: Raise your patient to a comfort position with a 15 degree electric backrest option available.

Application

Designed for heavy use and demanding environments.

Why Buy?

Having equipment that is robust and completely reliable is of the utmost importance when caring for larger or heavier patients. We have over-engineered the componentry strength in this design and our decision to use total stainless steel reinforces the lifetime of value that this bariatric shower trolley will give. The electric operation helps to safely lift 300kg without overload, whilst four twin wheels with brakes with wider base and top all increasing versatility in special needs.

Aquatuff Bari



Safe transfer

The drop sides fold underneath the top and the liner side lays down to provide gap free patient transfer.



Easy drainage

Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.



Backrest option

Raise your patient to a comfort position with a 15 degree electric backrest.



Length adjustment

Lock the removable end rails (on each end of the top) into two positions, allowing 120mm extra space for longer patients.

Specifications

Size: 1195 W x 1950mm including bumpers

Liner Inner (min) Base Size: 950 W x 1700 L x 180mm Deep

Standard Hose Length: 1100mm, custom lengths available

SWL: 300kg

Height Range: 680 to 900mm

Castor Size: Quality 125mm twin wheel castors with total lock and steer

Electric Backrest Option: 15 degree

Construction: Powder coated stainless steel

Product Code

Aquatuff Bari; MST-BARI

Electric backrest option - MST-SA-EBB

Aquatuff Electric

A stainless steel shower trolley with electrically operated 300kg SWL, wide top and backrest options.



Key Benefits

Safe & secure: The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfers.

Hygiene conscious: Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.

Custom adjustments: Raise or lower the height of your trolley to suit bed transfer or best working positions. You can also lock the removable end rails into two positions, allowing 120mm extra space for longer patients

Application

Designed for heavy use and demanding environments.

Why Buy?

Having equipment that is robust and completely reliable is of the utmost importance when caring for larger or heavier patients. We have over-engineered the componentry strength in this design and our decision to use total stainless steel reinforces the lifetime of value that this bariatric shower trolley will give. The electric operation helps to safely lift 300kg without overload, whilst four twin wheels with brakes, electric backrest and wider top options all increase versatility in special needs.

Aquatuff Electric



Standard design

The standard top has a full length dropside with an electric backrest raise option for extra comfort if required.



Safe transfer

The dropsides fold underneath the top and the liner side lays down to provide gap free patient transfer.



Easy drainage

Tilt the top 2 degrees and lower the drain hose tip to quickly remove waste water.



Height adjustment

Raise or lower the height of your trolley to suit bed transfers or best working positions.



Length adjustment

Lock the removable end rails (on each end of the top) into two positions, allowing 120mm extra space for longer patients.



Wide top option

Add a 100mm wider top for larger patients with wide shoulders.

Specifications

Size: 795 W x 1950mm including bumpers

Liner Inner Base Size: 550 W x 1700 L x 180mm Deep

Standard Hose Length: 1100mm

SWL: 300kg

Height Range: 680 to 900mm

Castor Size: Quality 125mm twin wheel castors

Construction: Powder coated stainless steel

Junior Size: 795 W x 1680mm

Junior Liner Inner Base Size: 560 W x 1450 L x 180mm Deep

Wider Comfort Option: 100mm

Product Code

Aquatuff Electric; MST-AQTE

Aquatuff Electric Junior; MST-AQTE-J

Electric backrest option (excludes Junior size) - MST-SA-EBA

100mm wider top option (excludes Junior size) - MST-SA-EWT

Change Table

A purpose built, non-slip change mat on a sturdy, height adjustable and safe patient platform.



Key Benefits

Safe & secure: The fold up dropsides and removable end rails ensure high levels of patient safety when required.

Customisable design: Tailor to your requirements with options such as electric high/low, 300kg SWL and wide tops.

Why Buy?

This change mat expands the use of the Aquarius trolley into a very safe, height adjustable change table with multi-position, removable end rails and full length dropsides that fold back underneath the trolley top for ample work clearance. The mat securely fastens underneath the trolley top to prevent sliding and movement. This trolley is designed for harsh use and able to be rigorously cleaned.

Change Table



Safe design

The fold up dropsides and removable end rails ensure a high level of patient safety when required.

Specifications

Size: 795 W x 1950mm including bumpers and 25mm high density foam core

Patient Surface Size: 600 W x 1820mm Long

Height Range: 60 to 900mm (depending on trolley model)

SWL: Hydraulic 180kg or electric 300kg

Castor Size: Quality 125mm twin wheel castors with total lock and steer

Options: Both junior and wider sized available



Customise

Tailor to your requirements with options such as electric high/low, 300kg SWL and wide tops.

Product Code

MST-SA-CTU

Custom Liner Colours

Customise the colour of your shower trolley liner from the standard Royal Blue.

Key Benefits

- Heavy duty washable, non-stick vinyl
- Ask for the available colours

Product Code

- MST-SA-CLC

Why Buy

There are a selection of light and dark colours available in the high grade vinyl used for the Aqua liners, that you can choose as an optional upgrade. Please contact us to discuss your requirements and view a current colour chart.



Royal Blue

