



Emergency Stretcher Beds



Emergency Stretcher Beds

In a trauma department, we understand that reliable equipment is critical because it increases patient throughput, improves operating costs and helps keep you in control. We provide emergency stretchers and recovery trolleys that work with your current staff systems and easily adapt to technology upgrades, so you're always covered.

Hospital emergency trolleys

Modsel's exclusive range of hospital emergency trolley designs continues to be a solution for healthcare and nursing professionals.



Contour Recline



Contour Energy



Contour Portare



Contour Jazz



Contour Classic



Contour Exami



Contour Heli-Transfer




Contour Portare-X




Contour Multi-X



Contour Barituff-X

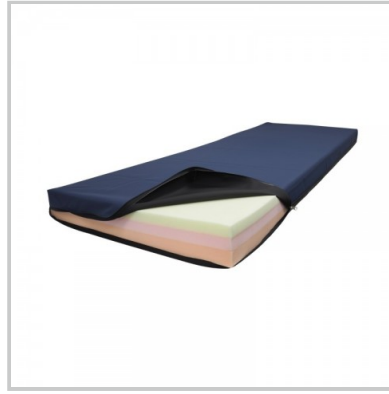

Stretcher Mattress

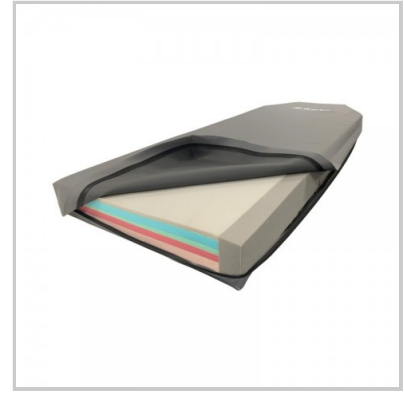

Premium Stretcher
Mattress



Orbit-Drive



Stretcher IV Poles



Oxy Holders



Dropdown Push Handles



Document Holder



End Rail



End Board



Dropdown Foot Board



Folding Monitor Board



Folding Monitor-Foot Board



Weigh Scales



Meal Tray

Contour Recline

A versatile chair to streamline medical examination, procedures and recovery across many healthcare facilities.



Key Benefits

- **Save Space:** Fit more chairs in the same space by increasing throughput with faster recoveries from patients sitting up right.
- **Hygiene conscious:** Smooth, gap free platform aids cleaning and helps prevent fluid ingress to meet the highest infection control standards.
- **Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.
- **Custom quality:** Manufactured in Australia with quality, care and customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance and fast guaranteed parts.

Application

Ideal for Day Surgery, Recovery, Transit Lounge, Fast Track ED & Cardiac

Why Buy?

Make better use of your valuable space and streamline your department with Modsel's compact procedure chairs. They are easy to move in and out of tight areas meaning a higher patient throughput and reduced waiting times. Patients can be positioned easily from sitting upright to a full recline, or lay completely flat with the stability of twin columns under the chair which can be used for CPR or specialised procedures. We've taken care with every aspect of design to ensure comfort and efficiency for both the patient and carer. Pop-up push handles ensure best operator positioning while preventing transfer movement to the patient.

Contour Recline



Low and safe

Excellent 460mm low height with fold-under drop-sides.

The ultra-low height is combined with drop-sides that glide down vertically under the chair top to provide safe access with no protrusions to catch on the patient's legs or clothes.



Easy egress

Forward tilt facilitates getting on and off.

For aged or disabled patients, a gentle incline and the long drop-side armrests may assist in safer transfer.



Lies flat

The backrest and leg rest move independently or together from chair to bed.

You now have the flexibility of sitting a patient up to almost 90 degrees or lying down completely flat.



Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



Smart transfer

Conveniently stored handles unfold for staff friendly transportation.

Lock in the double steer system, unfold the push handles and move forward effortlessly on the large castors. The unique handle design correctly positions the carer while preventing unwanted forces being applied directly to the patient.



Wide option

100mm wider top option available.

Change the seat width from 650mm to 750mm wide to easily accommodate larger patients. Use our range of colour coding choices if required to assist in quick identification.



Clear handset

Large simple functions with colour assistance.

Position the patient quickly and easily with the touch of a button - either by independently moving sections or applying one touch contouring.

Specifications

- **Size:** 780 x 1150mm overall (1750mm lying flat)
- **Seat width:** 650mm with 100mm wider option available with 100mm thick premium, removable mattress
- **Height Range:** 460 - 840mm with lie-flat and forward / reverse tilt
- **SWL:** 300kg
- **Castor Size:** Twin wheel 150mm castors with double steer
- **Package Includes:** Standard IV, vertical oxy holder, dropdown footrest & meal tray

Product Codes

- Contour Recline: MPC-C-RET
- Contour Recline Package MPC-C-RET-P
- 750mm wide top option: MPC-CA-WCT

Contour Energy

A transport chair with ZERO effort, 4-way power drive reducing the risk of strain during patient transport.



Key Benefits

- **Ergonomics & intuitive:** Drive effortlessly in any direction with only the slightest of touch, paired with perfect positioning of the drive controls for ease of use.
- **Easy access:** The large footplates offer generous support to your patients and lift quickly for clear egress.
- **Powerful movement:** A rotating electric drive wheel provides excellent traction, while center steering makes patient transport easier over longer distances.

Why Buy?

Take all the human effort out of pushing and maneuvering chairs with an intuitive and intelligent drive system designed for easy use by any operator. Natural movement controls the speed and direction of the chair with the ability to stop very quickly by pulling back. Help prevent the most common carer strain injuries by taking the back twist out of moving heavy weight forwards, backwards and sideways. Patients are positioned comfortably with a gentle recline and feet fully supported. We've taken care with every aspect of design to ensure comfort and efficiency for both the patient and carer. Customise for a wide range of healthcare areas with relevant accessories.

Contour Energy



Ergonomic

Drive effortlessly in any direction by only light touch.



Intuitive

Perfect positioning of the drive controls for the operator.



Support

The large foot plates offer generous support to the patient.



Easy access

The large foot plates lift quickly and easily for clear egress.



Full clearance

Complete frontal clearance by lifting and rotating foot plates outwards.



Removable armrests

The side armrests can be fully removed for multi direction egress.



Versatile

Use the storage shelf during transport or customise with accessories such as IV poles, oxy holders and seat belt.

Specifications

- **Size:** 750mm W x 1300mm L x 1180mm H
- **Seat Size:** 650mm W x 520mm D x 550mm H
- **SWL:** 300kg
- **Castor Size:** 150mm
- **Package Includes:** standard IV & vertical oxy holder

Product Code

- Contour Energy: MPC-C-ENT-EDU
- Contour Energy Package: MPC-C-ENT-EDU-P

Contour Portare

An all-purpose electric transport and recovery stretcher with excellent low height.



Key Benefits

- **Safer solution:** The 400mm low platform height of this stretcher takes away the requirement to use step stools or unnecessary heavy lifting.
- **Effortless control:** Sit and position your patient comfortably and then raise and lower them to the desired work height with the touch of a button.
- **Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Transfer stretcher suitable for most medical departments

Why Buy?

Helping patients on and off a general purpose stretcher has never been this easy or safe with a design that allows the mattress base down to 400mm, lower than a standard office chair height! Transport must always be considered and with our new E-Point system, a strong grip with the floor and solid, perfectly centered steer wheel ensures the stretcher always accurately tracks and turns. We bring Modsel stretcher design into department areas such as emergency, theatre and day surgery helping to maximise the number of working units available.

Contour Portare



Ultra-low height

Very safe and easy for patients to get on and off.

The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 830mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.



E-Point steering

A perfectly centred steer wheel ensures the stretcher always accurately tracks and turns.

Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction. Turning left or right from either end gives the same steering result.

Specifications

- **Overall Size:** 2135 x 815mm (965mm wide option)
- **Patient Surface:** 1900 x 680mm (830mm wide option)
- **Height Range:** 400 - 830mm
- **SWL:** 300kg
- **Castor Size:** 150mm twin wheel castors
- **Package Includes:** 100mm premia mattress, folding IV, vertical oxy, foot endboard, drop down push handles, emergency CPR backrest release, 2nd handset & USB port

Product Code

- Contour Portare: MPT-C-PE2
- Contour Portare Package: MPT-C-PE2-P
- 150mm Wider Top: MPT-CA-WCT

Contour Jazz

An Australian made transport and recovery stretcher that's a joy to use and designed for supreme patient safety and comfort.



Key Benefits

- **Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor
- **Safety and support:** The ultra-low 400mm height makes it safer and easier for patients to get on and off – no need for step stools or unnecessary heavy lifting.
- **Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.
- **Bright and light:** Available in a choice of four fun and colourful themes to brighten their day (and yours).
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for: Emergency, Theatre & Day Surgery

Why Buy?

Keeping patients safe, secure, and comfortable has never been easier with the Contour Jazz transport and recovery stretcher. With full, touch button electronic control and innovative E-Point steering system, the stretcher offers a smooth ride for patients and the kind of ease of operation and manoeuvrability that every caring professional craves. The Contour Jazz has been expertly designed to ensure you'll never miss a beat in even the most challenging environment or circumstances. Featuring a fast-action CPR backrest release, strong grip with the floor and an ultra-low height range that allows the mattress base down to 400mm. That's lower than a standard office chair height! Available in a range of colours and themes.

Contour Jazz



Ready for a range of accessories



Ultra-low height

Very safe and easy for patients to get on and off.

The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Music Theme



Sea Theme



Jungle Theme



Space Theme



Fixed Control Panel

A fixed control panel on the foot end of the stretcher prevents unwanted misuse.

Specifications

- **Overall Size:** 2135 x 815mm (965mm wide option)
- **Patient Surface:** 1900 x 680mm (830mm wide option)
- **Height Range:** 400mm-830mm
- **SWL:** 300kg
- **Castor Size:** 150mm twin wheel castors
- **Package Includes:** Handset and end controls, 100mm Premia mattress, emergency CPR back rest release, foot end board, drop down push handles, vertical oxy holder, folding IV, 16 riser dropsides and USB Port.

Product Code

MPT-C-JAZZ

Decal Codes

- MPT-CJA-MUSIC
- MPT-CJA-SEA
- MPT-CJA-JUNGLE
- MPT-CJA-SPACE

Contour Classic

The most flexible custom design platform available for a transport and recovery stretcher.



Key Benefits

- **Customisable design:** Choose from a variety of dropside options and electric or hydraulic versions available.
- **Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.
- **Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for Day Surgery, Recovery, Transit Lounge & Cardiac

Why Buy?

The most flexible platform of all our stretcher range. The twin column design allows for several height range choices - we can build in a low height range for patient egress care or higher height ranges for better equipment or surgeon access with less strain during procedures.

Tailor make the Contour Classic with shorter, longer, narrower and wider top sizes and select from a range of dropside styles to suit specialised procedures or the patient type you are working with. To simply put it, there is a high level of customisation possible with many accessory mounting options and solutions.

Contour Classic



Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 810mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



Easy customisation

Increase top width, change colours, mattress and dropside styles.

Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.



Ultra-low height

Very safe and easy for patients to get on and off.

The 440mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.

Specifications

- **Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)
- **Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)
- **Electric Height Ranges:** 440 - 810 (std) up to 540 - 910mm
- **Hydraulic Height Range:** 525 - 850mm
- **SWL:** 300kg
- **Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering
- **Electric Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard, drop down push handles, 2nd handset & USB port
- **Hydraulic Package Includes:** 100mm premia mattress, folding IV, vertical oxy, dropdown footboard & drop down push handles

Product Code

- Electric: MPT-C-CE
- Electric Package: MPT-C-CE-P
- Hydraulic: MPT-C-CH
- Hydraulic Package: MPT-C-CH-P

Contour Exami

An economical, customisable, easy to move examination table available with or without dropsides.



Key Benefits

- **Safe & low:** The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting.
- **Excellent height:** Electrically adjust the patient height up to 930mm helps to examine or treat your patient without having to stoop over, ensuring easy access and less stress.
- **Comfortable:** 100mm fluid proof Protran mattress with a welded, pressure relieving cover.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for medical clinics, examination rooms, schools, prisons & mining sites

Why Buy?

Helping patients on and off an examination table is easy and safe with a design that allows the mattress platform down to 400mm, lower than a standard office chair height! Uniquely, these stretcher examination tables are designed to be easily moved around and re-positioned due to their light-weight build and large castors. There are a range of accessories and customisations available to ensure the unit is designed to be priced and configured best to practically fit into your modern facility.

Contour Exami



Without dropsides

If you require a patient examination table, then this is worth considering. The standard Contour Exami without dropsides gives all the same functionality as an examination table, however has the additional value of colour choices and accessory mounting, while being very easy to move and reposition as required.



Ultra-low height

Make it easier and safer for patients to get on and off. The 400mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Excellent high height

Don't strain your back, raise a patient up to your best working height. Lifting the platform height up to 830mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release option available.



Head and foot tilt

A generous 15 degrees top tilt both directions. Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



Customise it your way

Upgrades such as wider top, dropsides, base cover, colours and accessories. This is a very flexible platform with excellent customisation available. Change the standard Contour Exami from a basic examination table into an economy patient stretcher.



Go anywhere

Change from the standard 125mm twin to 175mm single castors for situations such as outdoor. If there is a requirement to wheel over rougher concrete, sand or grass terrains, this option gives larger diameter rubber wheels with 50mm extra ground clearance under the stretcher.

Specifications

- **Overall Size:** 2135 x 815mm (965mm wide option)
- **Patient Surface:** 1900 x 680mm (830mm wide option)
- **Height Range:** 400 to 830mm
- **SWL:** 300kg
- **Castor Size:** 125mm twin wheel castors, 4 brakes and 1 steer
- **Includes:** 100mm Protran mattress, electric backrest that raises to 85 degrees, standard main power electric operation
- **Options:** dropsides, base cover, battery back-up with cable retractor, larger castors and 6 endrail colours

Product Code

- Contour Exami (without dropsides); MPT-C-EXAMI
- 150mm wider top; MPT-CXA-WT
- Dropsides; MPT-CXA-5RD
- Base cover with storage; MPT-CXA-BC
- Battery backup; MPT-CXA-BB
- CPR backrest release; MPT-CXA-BQR
- 175mm large castor option; MPT-CXA-175

Contour Heli-Transfer

A stretcher designed to reduce helicopter patient transfer effort outdoors.



Key Benefits

- **Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor.
- **Supercharged:** Keep things moving with long-life, cordless, fast-charge battery packs that work all day.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for patient transfers

Why Buy?

Safely and easily move a helicopter stretcher across onto this transfer stretcher, a platform designed to minimise effort whilst transporting patients over rougher outdoor surfaces. After benefiting from the smooth, flat bridge that lines up with the helicopter door, rapidly fasten the helicopter stretcher securely down with the four quick pull straps.

Reduce the trauma to patients and babies in cribs and the workload for carers whilst transferring via the large diameter castors and very stable stretcher.

Contour Heli-Transfer



Hydraulic option

Non-powered, manually operated stretcher option.

In some cases, such as small or remote facilities, or large high pressure departments, hydraulically operated stretchers may be more beneficial, with simple mechanical servicing and without the requirement of regular charging or power cords.



Orbit Drive Upgrade

Help prevent strain injury with this unique forward, back and sideways powered drive system.



RACQ Life Flight

Built in backrest with a special thin high density mattress and split dropsides.



Westpac

6 riser dopside and rubberised outdoor castor wheels.

Specifications

- **Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)
- **Patient Surface:** 1900 x 700mm (590mm narrow & 850mm wide options)
- **Electric Height Ranges:** 550 - 920mm
- **Hydraulic Height Ranges:** 530 - 860mm
- **SWL:** 300kg
- **Castor Size:** 200mm rubberised castors for outdoor shock absorption
- **Package Includes:** tie down straps, orbit drive, all-terrain castors, flat stretcher top with tie down straps and gap bridge, fully customised top surface option to suit special purpose & comes standard without backrest
- **Options:** Can be supplied with or without dropsides, emergency CPR backrest available

Product Code

- Electric: MPT-C-HE
- Electric Package: MPT-C-HE-P
- Hydraulic: MPT-C-HH

Contour Portare-X

An ultra-low electric trauma stretcher with totally unobstructed x-ray clearance.



Key Benefits

- **Unobstructed x-rays:** Supported by just four outside corners, our unique x-ray system gives total freedom of movement for cassettes - even with an electric backrest!
- **Safe & low:** The 430mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting.
- **Smooth & steady:** Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for ED & Imaging

Why Buy?

Helping patients on and off a trauma stretcher is safer and easier when the bed can be lowered down to 430mm, around the standard office chair height! This stretcher can feature our unique InfineX* system which allows totally unobstructed head to toe clearance for effortless and efficient x-ray cassette positioning. Smoothly moving patients helps improve their experience and our E-Point steering delivers a strong grip to the floor with a solid, perfectly centered wheel to allow the stretcher to always accurately track and turn. The Modsel x-ray stretchers are designed to be a reliable, cost-effective performer, no matter what the shift brings.

*Patent pending.

Contour Portare-X



Unobstructed x-ray tunnel

100% x-ray clearance of the entire body. Supported by just four outside corners, our unique patented InfineX system gives total freedom of movement for x-ray cassettes - even with an electric backrest! There are two styles available, another that allows the backrest to raise separately to the lower cassette support surface.



Ultra-low height

Safer and easier for unwell patients to get on and off. The 430mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Excellent high height

Quickly raise a patient up to the best working height. Lifting the platform height up to 880mm helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



Head and foot tilt

A generous 15 degrees top tilt both directions. Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



E-Point steering

A perfectly centred steer wheel ensures the stretcher always accurately tracks and turns. Operated by a very light touch pedal action, this wheel positively pre-loads onto the floor to ensure strong grip and rides up and down effortlessly over bumps and hollows to keep traction. Turning left or right from either end gives the same steering result.



Easy customisation

Increase top width, change colours, mattress and dropside styles. Every trauma department has their own unique requirements to be able to function safely and efficiently.



Cassette holder options

Holders can be produced with single or double sided handles and any depth or shaped rebates to suit all of the unique cassette sizes in use today

Specifications

- **Overall Size:** 2135 x 815mm (965mm wide option)
- **Patient Surface:** 1900 x 680mm (830mm wide option)
- **Height Range:** 430 - 880mm
- **SWL:** 300kg
- **Castor Size:** 150mm twin wheel castors
- **Package Includes:** 100mm premia mattress, folding IV, vertical oxy, foot endboard, emergency CPR backrest release, E-Point steering system, drop down push handles, 2nd handset & USB port
- **Options:** Unique InfineX x-ray system option

Product Codes

- Contour Portare-X: MPT-C-PEX
- Contour Portare-X Package: MPT-C-PEX-P
- 150mm Wider Top: MPT-CA-WCT
- X-Ray Cassette Holder: MPT-CA-XCH-D

Contour Multi-X

A trauma x-ray stretcher with ability to custom design and build to adapt to any emergency system.



Key Benefits

- **Smooth and steady:** An advanced E-Point steering system and solid, perfectly centred steer wheel ensure the stretcher accurately tracks and turns, while maintaining a strong grip with the floor.
- **Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for ED / Imaging

Why Buy?

This stretcher allows our unique InfineX* tracking which gives totally unobstructed head to toe clearance for effortless and efficient x-ray positioning, with an electric backrest that can raise separately to the cassette support surface. We can help you get the most out of your expensive x-ray systems by building to suit the brand and equipment in use. Tailor make the Contour Multi-X with height range choices and shorter, longer, narrower or wider top sizes. Put safety first and select from a range of droptop styles to suit the department requirements.

*Patent pending.

Contour Multi-X



Electric operation

Battery powered touch button control. Sit and position your patient comfortably and then raise them to the desired work height with ease.



Hydraulic option

Non-powered, manually operated stretcher option. In remote facilities, or large departments, hydraulic stretchers may be more beneficial, with simple servicing and without the requirement of regular charging or power cords. Includes a tracking digital cassette carriage and holder.



Electric standard

Standard powered stretcher in base configuration. Sit and position your patient comfortably and then raise them to the desired work height with ease.



Ultra-low height

Easier for patients to get on and off. The 545mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Excellent high height

Preserve your back and raise a patient up to your best working height. Lifting the platform height up to 915mm (plus an extra 100mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



Head and foot tilt

A generous 15 degrees top tilt both directions. Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



Easy customisation

Increase top width, change colours, mattress and dropside styles. Whether it is a specialised medical clinic or a department in a large hospital, every single customer has their own unique requirements to be able to function safely and efficiently. This stretcher has a longer, 7 riser PVC upgrade dropside.

Specifications

- **Overall Size:** 2135 x 815mm (705mm narrow & 965mm wide options)
- **Patient Surface:** 1900 x 680mm (570mm narrow & 830mm wide options)
- **Electric Height Range:** 545 - 915mm
- **Hydraulic Height Range:** 650 - 970mm
- **SWL:** 300kg
- **Castor Size:** 200mm castors and Ezi-Trak 5th wheel steering
- **Electric Package Includes:** 100mm premia mattress, folding IV, dropdown footboard, vertical oxy, drop down push handles, 2nd handset & USB port
- **Hydraulic Package Includes:** 100mm premia mattress, folding IV, dropdown footboard, vertical oxy & drop down push handles

Product Code

- Electric: MPT-C-XE-EB
- Electric Package: MPT-C-XE-EB-P
- Hydraulic: MPT-C-XH
- Hydraulic Package: MPT-C-XH-P
- X-Ray Cassette Holder: MPT-CA-XCH-D

Contour Barituff-X

A 500kg SWL extra-wide electric trauma stretcher designed for Bariatric patients.



Key Benefits

- **Purpose built:** Increased componentry strength in the core frame design operates with higher capacity columns to safely lift 500kg without overload.
- **Effortless control:** Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.
- **Custom quality:** Manufactured in Australia with speed, quality, and care, plus customisation capability.
- **Longest warranty:** Peace of mind with a lifetime of local support, easy maintenance, and fast guaranteed parts.

Application

Ideal for ED / Imaging and Bariatric Care

Why Buy?

Having equipment that is robust enough, that you can completely rely on, is the greatest challenge when caring for Bariatric patients. The Contour Barituff-X has increased componentry strength in the core frame design and operates with higher capacity columns to safely lift 500kg without overload.

The stronger and wider top comfortably caters for large patients, with a choice of mattress styles available to suit the type of departmental use. Stretcher stability is assured through an increased base size. Powered drive and weigh scales genuinely reduce manual handling risks to the carer.

This stretcher has been built for Bariatric patients.

Contour Barituff-X



Ultra-low height

Easier for patients to get on and off. The 600mm low platform height of this table takes away the requirement to use step stools or unnecessary heavy lifting. Sit and position your patient comfortably and then raise them to the desired work height with the touch of a button.



Excellent high height

Preserve your back and raise a patient up to your best working height.

Lifting the platform height up to 980mm (plus an extra 110mm for the mattress pad thickness), helps you to examine and treat your patient without having to stoop over, ensuring easy access and less stress. The backrest travels from 0 to 85 degrees with emergency CPR quick release.



Head and foot tilt

A generous 15 degrees top tilt both directions.

Position a patient how you need them or quickly lower their head down (and feet up) in an emergency.



Cassette holder options

Holders can be produced with single or double sided handles and any depth or shaped rebates to suit all of the unique cassette sizes in use today

Specifications

- **Overall Size:** 2135 x 1100mm
- **Patient Surface:** 1900 x 980mm
- **Mattress:** 110mm thick with 150mm thick option
- **Height Range:** 600 - 980mm
- **SWL:** 500kg
- **Castor Size:** 150mm twin wheel castors
- **Package Includes:** 110mm Premia Bariatric mattress, folding IV, weigh scales, vertical oxy, orbit drive, 2nd handset & USB port

Product Codes

- Contour Barituff-X: MPT-C-BEX
- Contour Barituff-X Package: MPT-C-BEX-P
- X-Ray Cassette Holder: MPT-CA-XCH-D

Stretcher Mattress

High grade mattress with excellent durability and pressure relieving comfort for patient stretchers.

Key Benefits

- Thicknesses that aid egress and imaging
- Protran mattress - single foam layer
- Premia mattress - 3 effective foam layers
- Premia Barituff mattress - 4 support layers (110mm thick 60 - 350kg, 150mm thick 45 - 400kg therapeutic support loads)
- Welded 'double coated' stretch cover
- Non-slip base design
- Permanent care label
- New technology fluid proof zip (excludes 75mm mattress)
- MADE IN AUSTRALIA

Product Code

- 100mm Protran Mattress; MPT-CM-100-PTP
- 100mm Premia Mattress; MPT-CM-100-PRP
- 110mm Barituff Mattress; MPT-CM-110-PRB
- 150mm Barituff Mattress; MPT-CM-150-PRB

Mattress Size



- Standard; 680 W x 1900mm L
- Narrow; 570mm W x 1900mm L
- Wide; 830mm W x 1900mm L
- Barituff; 980mm W x 1900mm L

Mattress Shape

- Square Ends (electric stretcher)
- Truncated End (hydraulic stretcher)

Why Buy

The Contour Premia mattress has been designed to optimize patient weight capacity and comfort by using a selection of quality foam layers to help in both preventing patients from bottoming and pressure reduction. The Contour Protran mattress is recommended for short stay patients with a single medium density foam section to facilitate special patient transfers or clear imaging.

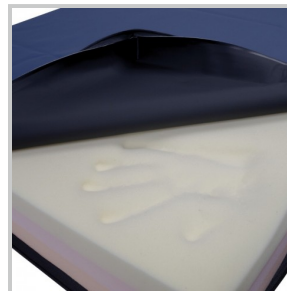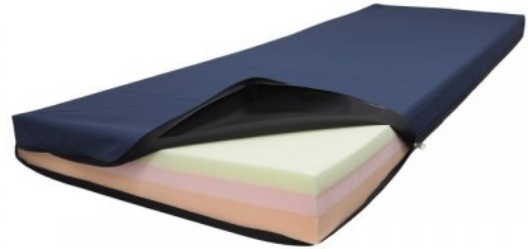

Pressure care design




Welded cover




Sealed zip



Care label

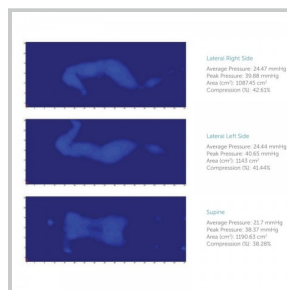

Pressure mapped

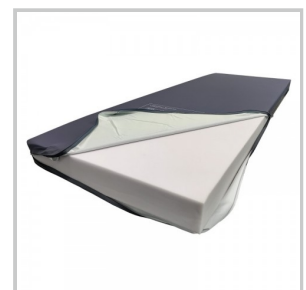

Protran

Premium Stretcher Mattress

High performance mattress with outstanding durability and patient support within demanding acute environments.



Key Benefits

- **Crucial support:** 5-layered high performance core for demanding environments.
- **Structural stability:** Integrated profile stabilizing sides and unique hinging system.
- **Fluid control:** Double coated, welded polyurethane cover with fluid proof zip.
- **Body distribution:** Superior 110mm mattress thickness with up to 350kg mass range.
- **Peace of mind:** Australian made plus offering a 5 year core foam and 4 year cover warranty.

Premium Stretcher Mattress



Stabilizing sides

High density bars down each side help to prevent roll off and assist the patient during egress by increased side strength of the mattress.




Premium pressure relief

The special combination of foam densities and layers provide superior patient support.




Profile cut sides

Strategically placed reliefs allow this thicker and denser mattress to conform to the stretcher surface contouring.

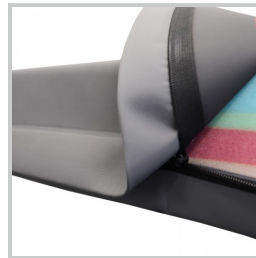

Fluid proof zip

The unique zip design seals when closed to resist fluid penetration, with the added protection of a premium waterfall flap.

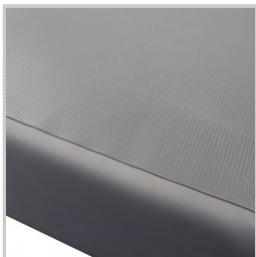

Non-slip base

A specially coated base fabric helps to prevent unwanted movement of the mattress and grip the stretcher surface without requiring mechanical fastenings. This allows quick access and fast cleaning of the patient surfaces.

Why Buy

The **Premia Forte** pressure relieving radiolucent support surface is our high spec version of the standard Contour Premia mattress. The specialised core structure provides best protection to patients at risk of injury without compromising support in crucial areas. A Premiflex stretcher cover provides ultimate patient comfort and protection. This mattress delivers an effective outcome - combining both high risk pressure care and longevity for demanding stretcher applications.

Product Code

- Truncated End (hydraulic stretcher); MPT-CM-110-PRF
- Square Ends (electric stretcher); MPT-CM-110-PRPF

Mattress Size

- Standard: 680 W x 1900mm L
- Narrow: 570mm W x 1900mm L
- Wide: 830mm W x 1900mm L

Orbit-Drive

Taking patient transport to a new level in carer welfare with zero effort, 4-way power drive.



Key Benefits

- Forward, back and sideways powered drive
- Intuitive height adjustable controls with emergency stop
- Powerful electric drive centre steer wheel

Orbit-Drive



Adjustable controls

Lower the drive controls down below the mattress height for enhanced patient access and specialised accessory use.



Adaptable

Install the power drive system on many of the Contour stretcher models



Powerful drive

An electric driven wheel pressured onto the floor that provides excellent traction and center steering.



Sideways movement

The unique rotating design allows the stretcher to be driven in any direction - forwards, backwards and sideways!

Why Buy

Take all the human effort out of pushing and jostling stretchers with an intuitive and intelligent drive system designed for easy use by any operator. Natural movement controls the speed and direction of the stretcher with the ability to stop very quickly by pulling back.

Help prevent the most common carer strain injuries by taking the back twist out of moving a stretcher forwards, backwards and sideways. Importantly, we haven't overlooked safety features such as emergency stop and system lock out, which are standard. Add this system to our traditional vertical column stretcher designs such as Contour Barituff, Classic, Deluxe and Multi-X to improve both standard and Bariatric patient transport.

Product Code

- MPT-CA-EDU

Stretcher IV Poles

Include a standard removable or fold down IV Pole with your stretcher.

Key Benefits

- A strong and safe hook design
- Light and easy 'lift out' auto-lock hanger
- Non-rotational design to prevent tube tangling
- High strength locking for heavy loads
- Tough construction from stainless steel and nylon
- Easy 'pull-down' release from any angle
- Large 28mm diameter stand body for ease of pump mounting
- Friction retainer mount for effortless use
- Removable and permanent fold down options
- 705mm up to 1135mm total height
- 10kg weight capacity, 5kg per hook
- Only 1kg total weight

Product Code

- Standard IV Pole - MPT-CA-IVP
- Folding IV Pole - MPT-CA-FIV

Why Buy

Constructed tough from stainless steel and nylon, these clever non-rotational IV poles are designed to withstand the harshest use and make daily life easier by helping to prevent the tangling of critical soft tubing and cords. Simply lift out the hanger to any height and it automatically locks into an infinite position. Pull the lock body down from any direction to release and lower. The standard removable IV fits into all four corners of a stretcher top and stores underneath in the base if there is provision. The folding IV fits onto either end of the stretcher top and can be installed to fold down either to the left or right.



Standard IV storage



Folding IV Pole

Oxy Holders

A selection of coated stainless steel oxy bottle holders that can be mounted vertically or horizontally.

Key Benefits

- Tough, powder coated stainless steel basket construction
- Vertical basket size for up to 120mm diameter bottle; 320mm deep x 150mm wide
- A range of vertical mount positions - low, high with swivel or 45 and 90 degree fixed angles
- Horizontal basket size for up to 140mm diameter bottle; 600mm long x 180mm wide
- Horizontal basket can vary in hanging height depending on stretcher models; typically 280mm

Product Code

- Vertical Rail Mount - MPT-CA-VO2
- Vertical Corner Mount Swivel - MPT-CA-VCO2
- Vertical Corner Mount Fixed 45 - MPT-CA-VCO2-45
- Vertical Corner Mount Fixed 90 - MPT-CA-VCO2-90
- Vertical Corner High Mount Swivel - MPT-CA-VCH-S
- Vertical Corner High Mount Fixed 45 - MPT-CA-VCH-45
- Vertical Corner High Mount Fixed 90 - MPT-CA-VCH-90
- Horizontal Oxy Holder - MPT-CA-HO2

Why Buy

Due to so many wide and varied departmental requirements for specific oxy bottle sizes and positioning, we have developed a range of options. Larger bottles can be laid horizontally to ensure they are kept out of the way and smaller bottles are commonly mounted vertically. Some prefer 'fixed', non-moving positions and others require 'swinging' options that allow the bottles to be moved aside if they become an obstruction during transport. There is also the option to support bottles vertically both above and below the patient surface.



Vertical Corner Mount



Vertical Corner High Mount



Horizontal Oxy Holder



Horizontal Oxy Holder

Dropdown Push Handles

Strong fold-away handles with rubber grip for pushing the trolley.

Key Benefits

- Strong stainless steel construction
- Lift up and fold down
- Tough hygienic hand grips
- Tall head end handle option - 400mm high x 600mm wide (350mm between handles)
- Foot end handle option - 160mm high x 600mm wide (350mm between handles)

Product Codes

- Tall head end handles; MPT-CA-TDPH
- Foot end handles; PT-CA-DPH

Why Buy

To allow space for the many removable and common accessories used at the foot end of the stretcher, the handles are shorter to fold completely away with no obstruction to the accessory mounting positions. There is an option for a tall version of these handles to be mounted at the head end of the stretcher which provide high pushing points and ample clearance from the moving backrest.



Head end handles folded



Foot end handles



Foot end handles folded

Document Holder

Strong powder coated wire basket that is hung or mounted for document storage.

Key Benefits

- Ultra-tough, powder coated, 5mm steel wire construction
- External size; 390mm wide x 80mm deep x 230mm high
- Internal size; 380mm wide x 60mm deep x 225mm high
- Hang on the inside or outside of a stretcher end rail
- Wall and accessory mounting options

Product Code

- MPT-CA-DCH

Why Buy

Document holders can be used in a variety of ways for many different storage requirements. In the past we have mounted holders on either end or in special places around a stretcher, off accessories such as the dropdown foot board and even on the walls around the stretcher holding bays. We can customise sizes and designs and will work with you to ensure you can maximise department efficiency.



Removable holder

End Rail

An end rail that can help to retain the mattress, patient or be used as a push handle.

Key Benefits

- Strong, 25mm diameter coated stainless steel rail
- 600mm wide x 200mm high
- Fits into foot end or optional head end accessory sockets

Product Codes

- End rail; MPT-CA-FER
- Head end accessory sockets; MPT-CA-ASH (pair)

Why Buy

Quickly drop this rail into the accessory sockets (optional accessory on the head end of a stretcher, excludes Contour Vertex) and use it as a higher strong point to move the stretcher. This rail has a variety of additional uses such retaining a mattress or pillow from sliding off or a bar for a patients feet to push against. These rails can be used on either end of the stretcher or both ends together.



Head end option



Both ends

End Board

An end board that can help to retain the mattress, patient or be used as a higher pushing position.

Key Benefits

- Strong, 25mm diameter powder coated steel rail
- Tough, hospital grade solid plastic infill
- 600mm wide x 345mm high
- Fits into foot end or optional head end accessory sockets

Product Codes

- End board; MPT-CA-FRTI
- Head end accessory sockets; MPT-CA-ASH (pair)

Why Buy

Quickly drop this board into the accessory sockets (optional accessory on the head end of a stretcher, excludes Contour Vertex) and use it as a higher strong point to move the stretcher. This end board has a variety of additional uses such as retaining a mattress or pillow from sliding off or a flat surface for a patient's feet to push against. These boards can be used on either end of the stretcher or both ends together.



Head end option



Both ends

Dropdown Foot Board

Padded fold-down foot board to increase the length of the patient surface.

Key Benefits

- High density foam covered in welded stretch material
- Tough board construction in a one-piece hospital grade plastic
- Stainless steel lift and lock folding hinge
- 530mm wide x 330mm high (250mm high cushion)
- Adds an extra 310mm to the length of a mattress

Product Code

- MPT-CA-DDF

Why Buy

Padded for patient comfort, the removable dropdown foot board has molded handles for pushing the stretcher and with a simple 'lift and lay' action, folds down to quickly increase the length of the mattress area.



Fold down

Folding Monitor Board

A strong frame with push handles and folding board with an adjustable tie strap for retaining a monitor.

Key Benefits

- Powder coated steel frame
- Cross bars for patient feet protection
- One-piece hospital grade plastic board
- 50mm wide adjustable tie strap
- 450mm wide x 400mm deep tray surface
- 600 wide x 470mm high from mattress base
- 15kg weight capacity

Product Code

- MPT-CA-FMB

Why Buy

Add this removable monitor board accessory to the foot end of a stretcher to provide a solid push handle and fold-away board for securely mounting a monitor. The board hangs vertically out of the way and can be easily rotated outward and over to a stable, internal, horizontal rest position.



Fold over to a table



Writing surface option



Writing surface extended

Folding Monitor-Foot Board

A strong frame with both a folding foot and monitor board with an adjustable tie strap for retaining a monitor.

Key Benefits

- Powder coated steel frame
- One-piece hospital grade plastic board construction
- 600 wide x 410 deep x 540mm high overall size
- 520mm wide x 380mm deep monitor tray
- Monitor tray is 540mm high from mattress base
- 15kg weight capacity
- 50mm wide adjustable tie strap for monitors
- 500 wide x 250mm high foot board
- Adds 320mm to the length of the mattress

Product Code

- MPT-CA-FMFB

Why Buy

Add this removable monitor board accessory to the foot end of a stretcher to provide a solid push handle and fold-away board for securely mounting a monitor. The board hangs vertically out of the way and can be easily rotated outward and over to a stable, internal, horizontal rest position. Padded for patient comfort, the dropdown foot board folds with a simple 'lift and lay' action to quickly increase the length of the mattress area.



Fold over to a table



Fold down foot board



Writing surface option



Writing surface extended

Weigh Scales

Add a weigh scale system to Contour Stretchers for ease of patient measuring and tracking.

Key Benefits

- 200g accuracy
- Auto off with weight memory function
- Freeze weight to allow for additional accessories
- Excludes Contour Portare stretcher models

Product Code

- MPT-CA-WS



Easy to use

Meal Tray

Tough tray constructed from fluid proof board with a patient comfort cutaway.

Key Benefits

- Clips onto the stretcher dropside rail
- Board sizes to suite narrow, standard and wide stretcher tops
- Very tough, easy to clean fluid proof design

Product Code

- MPT-CA-MLT

Why Buy

Quickly add this compact meal tray which can be supplied to clip onto the stretcher dropside. The tray is constructed of a tough, one piece medical grade board that is very easy to clean. There is a recess to ensure patient comfort. This tray can be customised to any shape or size if required.



Notes

A series of 15 horizontal dashed lines for taking notes.

Notes

A series of horizontal dashed lines for writing notes.





MODSEL

modsel.com ▶