



Cleaning Instructions

Cleaning is always a concern. It must happen to control infection and cross contamination. However, ongoing changes in technology and supply can affect the life of your supporting equipment.

We continue to develop our products and use materials or coatings that will withstand the rough and aggressive chemical used in general cleaning programs. Our range of medical patient handling trolleys, chairs, drawers and mobile lockers have all been carefully designed with the ease of cleaning, safety and longevity in mind.

Cleaning Instructions

- Nonabrasive disinfectants are recommended in a diluted solution.
- Treat specific stains promptly to ensure complete removal. Wipe clean from outside to inside.
- Avoid harsh abrasives or high-heat soaks. Use a mild proprietary vinyl cleaner.
- Machine washing and autoclaving is not recommended.
- DO NOT use bleach or use caustic, caustic, abrasive or alkaline cleaners, bleach or any proprietary cleaners containing solvents.

DAMAGE CAUSED BY THESE PROCESSES ARE OBTAINABLE AND LOSS VOID WARRANTY

Wipe off cleaning product residue with a clean, soft cloth and rinse with fresh warm water after use. In particular cases manufacturers should be advised to read the particular surface description in advance or to contact us in case of any doubts.

MODSEL Page 1/1



Trusted products made for trusted people

15 Year Warranty

Designed by us, built-made with care by us and 100% backed by us... it gives us the confidence to always offer you the longest warranty.

We warrant in the event that:

- Any fabricated structure failure is found within 15 (fifteen) years from the date of invoice due to faulty workmanship.
- A part or parts of plastic not covered by the above warranty, are found within (5) five years from the date of invoice to have a structural defect.
- All other parts not covered by the above warranty are found within (2) two years from the date of invoice.
- The structure materials are found to have a manufacturing defect within (2) of invoice to have defect. This includes faulty cover material. Faulty floor casting and base. Failure of any internally support moving parts such as bearings, castors, hydraulic rams, gas struts, pistons etc.
- Any internally support moving parts such as bearings, castors, hydraulic rams, gas struts, pistons etc. from the date of invoice to have a structural defect.
- The purchase, with a 15 (fifteen) year, of discovery of the defect or defect, given written notice to Modsel or their distributor or agent, and:

The defect or defects are not due to:

- Cosmetic issues such as fading, staining, discoloration.
- Lack of maintenance as per the terms and conditions.
- Accidents, negligence, abuse, accidents, improper installation or normal wear and tear.
- Conditions such as close proximity to salt, experience or cold temperatures.
- Non-adherence to cleaning and care instructions.
- Removal of fluid substances and spillage.
- Incorrect usage of stored parts after the Cleaning Instructions.

Modsel will:

At our option, repair or replace the part or parts found within our reasonable judgement to have been defective under the warranty above, should your structure ever require attention due to a failure of any original purchase.

MODSEL Page 1/1



Trusted products made for trusted people

User Guides



User Guides

For things to run smoothly, you need to be informed. We have brought together some popular and useful information to ensure you know what you need to, when you need to.



Cleaning Instructions



15 Year Warranty



Cleaning Instructions

Cleaning is always a concern. It must happen to control infection and cross contamination, however ongoing changes in technology and supply can affect the life of vital supporting equipment.

We continue to develop our products and use materials or coatings that will withstand the tough and aggressive chemicals used in present cleaning programs. Our ranges of Modsel patient transport stretchers, chairs, showering and rounds trolleys have all been carefully designed with the ease of cleaning, safety and longevity in mind.

Cleaning Instructions

- Nonphenolic disinfectants are recommended in a diluted solution.
- Treat specific stains promptly to ensure complete removal, wipe stain from 'outside in' to avoid spreading.
- With difficult mattress or bath liner stains, use a mild proprietary vinyl cleaner.
- Machine washing and autoclaving is not recommended.
- DO NOT tumble dry, use phenolic disinfectants, abrasive or aqueous cleaners, steel wool or non-proprietary cleaners containing solvents.

DAMAGE CAUSED BY THESE PROCESSES ARE IDENTIFIABLE AND MAY VOID WARRANTY

Wipe off cleaning product residue with a clean, soft cloth and rinse with fresh warm water immediately after use. If particular cleaner/disinfectant is not listed we advise to test the particular surface disinfectant in advance or to contact us in case of any doubts.

Approved Cleaning Products

Actichlor Plus	2000ppm
Arjo Disinfectant	According to manufacturer's spec
Asepti Active Wipes	Ready to use
Chlor-Clean	1000ppm (1 tablet per 1 litre water)
Clinell Hand & Surface Sanitiser	According to manufacturer's spec
Clinell Universal Sanitising Wipes	According to manufacturer's spec
Clinell Detergent Wipes	According to manufacturer's spec
Clinell Sporidical Wipes	According to manufacturer's spec
Disinfecting Wipes	According to manufacturer's spec
Ecolab D4A	According to manufacturer's spec
Isopropyl Alcohol 70%	Ready to use
Iso Wipes (Kimberly-Clark)	Ready to use
Jumbo Liv-Wipes	According to manufacturer's spec
Lemex	According to manufacturer's spec
Mediwipes Disinfection (SL)	According to manufacturer's spec
Neutral Detergent Wipes (RHS)	According to manufacturer's spec
Sterrimatt (Pure) Foam	Ready to use
Super Sani Cloth Germicidal Disposable Wipes (PDI)	According to manufacturer's spec
Tuffie 5 Cleaning Wipes	According to manufacturer's spec



15 Year Warranty

Designed by us, hand-made with care by us and 100% backed by us... it gives us the confidence to always offer you the longest warranty.

We warrant in the event that:

- Any fabricated structure failure is found within (15) fifteen years from the date of invoice due to faulty workmanship.
- A part or parts of plastic not covered by the above warranty, are found within (5) five years from the date of invoice to have a structural defect.
- All electric parts not covered by the above warranty are found to have a manufacturing defect within (2) two years from the date of invoice.
- The stretcher mattress not covered by the above warranty, is found within (2) two years from the date of invoice to have defect. This includes faulty cover material, faulty seam welding and foam failure.
- Any externally supplied moving parts including springs, castors, hydraulic rams, gas struts, pivots etc not covered by the above warranty, are subject to third party warranties or found within (1) one year from the date of invoice to have a structural defect.
- The purchaser, within (14) fourteen days of discovery of the defect or defects gives written notice to Modsel or their distributor or agent, and:

The defect or defects are not due to:

- Cosmetic issues such as fading, crazing, discoloration.
- Lack of maintenance as per minimum requirements.
- Vandalism, negligence, abuse, accidents, improper installation or normal wear and tear.
- Products tampered with or altered, modified or repaired by anyone when not approved by Modsel.
- Conditions such as close proximity to salt, extreme hot or cold temperatures.
- Non-adherence to cleaning and care instructions.
- Permeation of fluid substances into foam.
- Incorrect usage of electric parts as per the Operating Directions.

Modsel will:

At our option, repair or replace the part or parts found within our reasonable judgement to have been defective in workmanship or materials. Should your Stretcher ever require attention due to a failure as defined under the warranty above, please [contact us](#) or your local distributor from whom your Stretcher Chair was originally purchased.

